ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use

INTENDED USE/INDICATIONS

All indications requiring immobilisation of the wrist and thumb, such as: sprains, strains, trauma, tendinopathy, rhizarthrosis, carpal tunnel syndrome, post-trauma, pre- or postoperative immobilisation.

CONTRAINDICATIONS

• None

WARNING & PRECAUTIONS

We recommend that the first application be made under supervision of a health care professional. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.

APPLICATION INSTRUCTIONS

- Loosen straps and slip hand into brace
- Place the thumb in the corresponding opening at the top of brace
- Tighten and secure the middle strap first, using the contact closure tab.
- Then tighten and secure the two other contact-closure straps.
- Wrap and secure thumb strap.

Straps are malleable and may be contoured to fit individual needs. Adjust the tightness of the straps according to compression and comfort needed.

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

Aluminium, Polyester, Polyamid, Polyurethane, PVC, Cotton, Polypropylene

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2018 DJO, LLC
MDS5 GmbH