
442 - Supershoe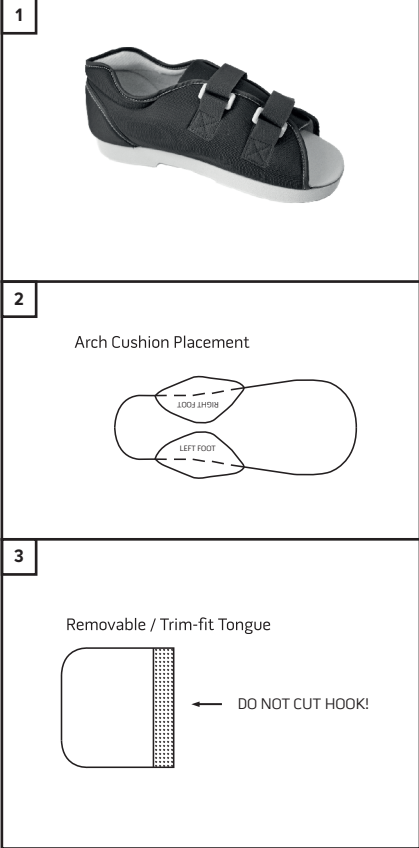

ENGLISH

Product Description

An adjustable medical shoe suitable for post-operative healing and injuries of the foot. A padded nylon post-op shoe with rigid non-slip semi-flexible sole and return loop D-Ring buckles.

Usage Information (See diagram)

- Unfasten both strap closures. Place foot into the shoe, ensuring heel is located to back of shoe. Re-fasten both straps by returning through the D-Rings and securing for a snug but comfortable fit. Trim the straps if required.
- A pressure sensitive longitudinal arch support is included for comfort.
- The removable tongue can be positioned anywhere on the inner for comfort. Do not cut hook.

Sizes and Fit

Size	Sole Length x Width
Extra Small	22cm x 7cm
Small Women's	24cm x 8cm
Medium Women's	25cm x 9cm
Large Women's	27cm x 9cm
Small Men's	28cm x 10cm
Medium Men's	29cm x 10cm
Large Men's	30cm x 10cm
Extra Large	32cm x 11cm

Indications for Use

- Accommodate bandages or swelling.
- Foot Surgery.
- Broken toe management.
- Metatarsal injuries.
- Soft tissue procedures.
- Management of diabetic foot.
- Post-operative recovery.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
 - Follow all instructions to ensure proper performance of the device.
 - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
 - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
 - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
 - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
 - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash shoe in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Composition

Nylon, PU, Polyester, Cotton, TPR, EVA.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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