ADHESIVE PLASTER
Instructions For Use

Description

Moleskin is a thin but heavy cotton fabric. It's soft on one side and has a sticky adhesive backing on the other. The product does not contain natural rubber latex. It is provided non-sterile.

Indications for Use

Use Moleskin to prevent a blister that has already formed from getting worse or popping open before you can treat it properly.

Contraindications

Diabetics and those with poor circulation should consult a physician before using the product.

Precautions & Warnings

- Avoid tearing the skin that forms the top of the bubble off as the flesh underneath will be much more sensitive.
- If the blister must be popped or pops on its own, leave the skin on for the remainder of the run.

Directions for Use


General Instructions:

- Irritated Patches (Use moleskin on a place where you can feel that you will soon have a blister.)
- 1) Cut a piece of moleskin the size of the affected area and place it firmly on your skin.
 - 2) Apply multiple layers of moleskin to the area to prevent chafing. Use enough so that your foot will not rub against your shoe.
 - 3) Cut a piece of athletic tape long enough to wrap around your foot.

Blisters:

- 1) Cut a piece of moleskin roughly the size of the affected area (figure 1).
- 2) Fold the moleskin in half (figure 2).
- 3) Cut a hole in the moleskin that is about the size of the blister (figure 3).
- 4) Unfold the moleskin and place it over the blister, with the blister through the hole. This cushions the blister but does not cover it (figure 4-5).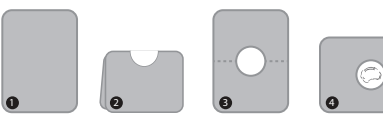

Open Blisters:

- (Use moleskin on a blister that has already opened.)
- 1) Clean the area if you can, using water or antiseptic wipes.
 - 2) Cover the area with clean gauze. Cut the gauze down to size as extra gauze may chafe.
 - 3) Cut a piece of moleskin that will cover the entire blister and gauze pad. Apply it to the foot.

Removal/Disposal

- Moleskin may be removed by pulling off of the skin or by loosening with sterile water or saline.
- Dispose of product per your facilities protocol.

Storage

For best results, store moleskin in original packaging at room temperature until use. Avoid excessive heat and humidity. Refer to the label on each box for the expiration date.

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