CE

**AFH TheraPIE Knete**<sup>®</sup> *Light*

DE

EN

FR

ES

IT

NL

DE

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## AFH TheraPIE Knete® *Light*

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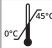
# Instructions for use

## AFH TheraPIE Knete® Light

	Follow instructions for use	<b>MD</b>	Medical device
	Application only for one patient		Before and after use wash your hands
	Not for children under 3 years		Not suitable for eating
	Protect from moisture		Protect from sunlight
	Latex free		Permissible temperature range

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**Intended use:**

This medical device is a latex-free therapeutic plasticine which is used to improve movement and function of the neuromuscular circuit through special finger and hand exercises. It is different from conventional therapie putty due to its low weight and is much softer in consistency with a lower viscosity.

These properties give AFH TheraPIE Plasticine® Light a larger volume. This means that even larger hands can work with a smaller number of grams. Due to its lower viscosity and softer consistency, strengthening exercises can be carried out in a way that is gentle on the joints. This makes AFH TheraPIE Knete® Light particularly suitable for sensitive patients and for a wide range of joint diseases (e.g. osteoarthritis) and neurological diseases (e.g. mild forms of stroke).

6 different resistances in 6 different colors offer an extensive spectrum in the selection of the optimal resistance or the appropriate amount of exercise. This is a welcome addition to the AFH therapy putty family.

**AFH TheraPIE Knete® Light** also stimulates blood circulation and simultaneously supports intramuscular and psychogenic stress reduction.

**General instructions for use:**

- It is used to promote finger and wrist mobility and helps to optimize the neuromuscular control circuit.
- Due to its joint-friendly properties, it is an optimal medium for various joint diseases from osteoarthritis to rheumatoid arthritis (in the early stages). It improves mobility and maintains everyday strength in these conditions.
- It promotes fine motor and sensorimotor skills.
-

- It supports the rehabilitation of mobility, e.g. after operations (note - after medical consultation and approval) and is used for general strengthening of the hand and finger joints using 6 degrees of firmness from "soft" to "extra firm".
- Depending on the exercise, **AFH TheraPIE Knete® Light** is available in the sizes 15 g (e.g. fingers), 85 g to 300 g (e.g. finger, hand and wrist) and 1.5 kg for self-fillers.

### **Indication:**

- Arthrosis of the finger and thumb joints / wrist
- Rheumatoid arthritis in the early stages
- Neurological diseases (after approval by the doctor or therapist)
- Muscle strengthening after operations (after approval by the doctor or therapist)
- Polyneuropathies
- Sensorimotor disorders
- Stress reduction and general muscle strengthening at work and in everyday life

### **Contraindication:**

Diseases of the skin of the hand and finger/thumb joints (e.g. neurodermatitis, open wounds, advanced rheumatoid arthritis, etc.): Neurodermatitis, open wounds, advanced rheumatoid arthritis, etc.).

### **Risks of use / Important information:**

- Use **AFH TheraPIE Knete® Light** as indicated.
- Simultaneous use with other products only after consultation with your doctor or therapist.
- Do not make any changes to the product.
- Do not use on open wounds in the hand and finger/thumb area.
- No reuse - this aid is intended for the care of a patient.

- Caution: not suitable for consumption.
- Not suitable for children under 3 years of age.
- Wash hands before and after use.
- Do not apply to clothing or other textiles.
- If putty has been placed in the eye, wash the eye thoroughly with water. If necessary a doctor if necessary.
- In case of pain, redness, swelling and other problems, stop the exercise frequency or the entire exercise program.

**Application instructions:**

Discuss the indication, application and duration of **AFH TheraPIE Kne-te® Light** with your prescribing doctor or therapist. The instruction will be given by the specialist staff from whom you received the medical device.

## Application examples

Finger flexion

Fingertip grip

Finger adduction

Palm ball rolling

Bilateral hand rolling

Hand pulling

**Storage:**

Due to the viscous respectively kinetic flow behavior of **AFH TheraPIE Knete® Light** and to prevent contamination, the TheraPIE Knete Light should always be stored in the sealed container at room temperature.

**Material composition:**

Polyisobutylene (20%), kerosene oil (18%), calcium carbonate (50%), silicon dioxide (2%), silicone oils (5.9%), acrylate polymers (3%), colorants (1%), boric acid (0.1%).

**Special aspects:**

**AFH TheraPIE Knete® Light** is vegan, gluten-free and latex-free!

**Cleaning instructions:**

The **AFH TheraPIE Knete® Light** cannot be cleaned. Please wash and disinfect your hands thoroughly before and after use.

**Warranty:**

The statutory provisions of the country in which you purchased the product apply to the purchased product. Please contact your specialist dealer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been sufficiently observed, the warranty may be impaired or invalidated. Warranty is excluded in the event of improper use, failure to observe the risks and instructions for use and unauthorized modifications to the product.

**Service life / service life of the product:**

The service life of the **AFH TheraPIE Knete® Light** is determined by natural wear-and-tear when handled properly and in accordance with the application. At the end of treatment, therapy or customized training, the TheraPIE Knete Light must be disposed of properly.

**Obligation to report:**

If there is a serious deterioration in your state of health when using the medical device, please report this to your specialist dealer or to us as the manufacturer or to the BfArM (Federal Institute for Drugs and Medical Devices).

If you are located in Switzerland, please contact our Swiss authorized representative at: [info@fg-health.ch](mailto:info@fg-health.ch) or Swissmedic directly if you have a complaint.

**Disposal:**

At the end of its service life, the product must be disposed of in accordance with local regulations.

**Declaration of conformity:**

We confirm that this product complies with the requirements of REGULATION (EU) 2017/745 of the European Parliament and of the Council.

---

## AFH TheraPIE Knete® Light

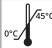
The AFH TheraPIE Knete® Light is available in the sizes 15 g, 85 g, 300 g and 1.5 kg and in 6 colors with different properties / resistance.

Colors	Weight
	15 g / 85 g 300 g / 1,5 kg

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