DANISH

Brugsanvisning

Fryser:

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Instructions for Use

Freezing:

- Place Polar Ice Plus Glove in the freezer for 1 ½ to 2 hours. Optimum freezer temperature is 0 degrees to 14 degrees Fahrenheit (-18 degrees to -10 degrees Celsius).

Freezing Instructions for Use:

1. Apply Polar Ice Plus Glove over surgical dressing immediately following surgery, strain or injury
2. For initial application, wear ice therapy wrap for up to 45 minutes over soft dressing or lightweight cloth. If applied directly to skin, Polar Ice Plus Glove should be worn about 20 minutes.
3. Reapply as needed for pain and swelling.

Heating:

- Place Polar Ice Plus Glove in the microwave for 30 seconds (1200-watt microwave). Gently touch the glove to test the temperature. Microwave in additional 10 second intervals to reach desired temperature. (Monitor the product while heating – remove if any product deformities occur)

Heating Instructions for Use:

1. Apply Polar Ice Plus Glove over surgical dressing immediately following surgery, strain or injury
2. For initial application, wear Polar Ice Plus Glove for up to 45 minutes over soft dressing or lightweight cloth. If applied directly to skin, Polar Ice Plus Glove should be worn about 20 minutes.
3. Reapply as needed for pain and swelling.

Hospital Patients:

- Polar Ice Plus Glove should be applied immediately following surgery to reduce pain and swelling if using cold application.
- Continue to apply until dressing is removed.
- Following removal of dressing, apply Polar Ice Plus Glove as needed.

Care:

- Hand wash and air dry.
- Polar Ice Plus Glove can be stored indefinitely in the freezer for immediate availability.
- For shorter duration cold therapy the unit may also be refrigerated.
- For transporting to athletic events, a cooler with ice maintains the frozen units.
- Always heat from room temperature– never frozen

Warning:

- Some individuals are sensitive to cold and/or heat.

- Cold therapy may cause frostbite if used improperly. Discontinue use and call your physician if skin becomes discolored or fails to regain normal color within 20 minutes following use.
- Overheating or prolonged application may cause injury.
- For extreme, chronic, or prolonged pain contact physician.

Brownmed warrants this product to be free of manufacturing defects

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