


# Plastic Angled Goniometer

## User Manual

### Intended Purpose

The Baseline<sup>®</sup> Plastic Angled Goniometer is used to measure joint range-of-motion (ROM).

**Use:** Place goniometer near joint to be measured; turn goniometer arms until pointer arrow on the arm is aligned with 0 on the scale; take joint through its range and adjust goniometer arms accordingly; the new degree the pointer arrow is aligned with is the angle measurement.

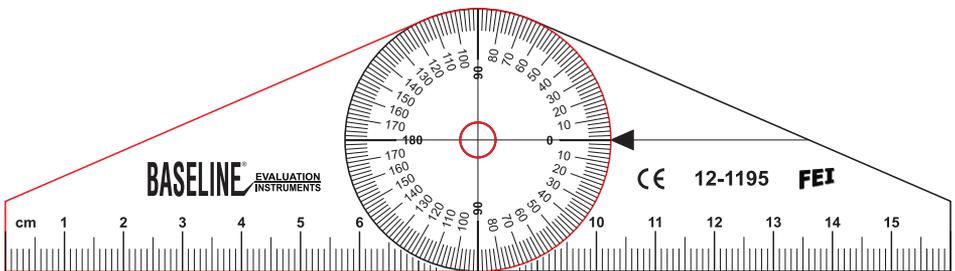

### WARNING:

- Verify calibration before each use
- Clean device between patients
- Zero device before measurement
- Apply consistent gentle pressure

### Measurement Tolerance

This device measures inches and centimeters with a tolerance of  $\pm 0.04$  inches ( $\pm 1$  millimeter) and degrees with a tolerance of  $\pm 1$  degree.

### Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.