
MECRON X-patch

EN Cross Tape
DE Gitter-Tape
ES Tape en rejilla
FR Tapes croisées
IT Cross tape
NL Rastertape
DA Gittertape
SV Gittertejp
PL Taśma siatkowa
SL Mrežast trak
HR Mrežasta traka
PT Fita em grade

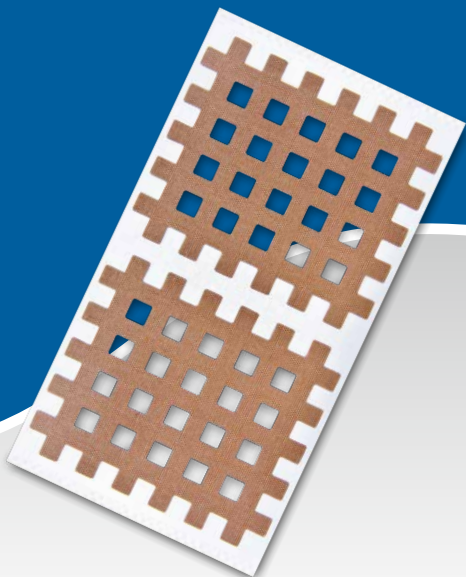MECRON

MECRON X-patch | Cross Tape

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product on injured, irritated, or damaged skin.

This product is a single-use product.

Purpose

The cross Acupuncture Tape is for the treatment of pain, trigger or acupuncture points.

Indications / Contraindications

Indications

- > Treatment of trigger points
- > Treatment of pain points and acupuncture meridians
- > Scar tissue release

Contraindications

- > Large skin injuries
- > Open wounds with risk of infection
- > Uncertain diagnoses
- > Allergic skin reactions to the tape material

Design features

Elastic tape in a grid pattern

EN Use

- > To begin with, obtain a clear picture of the condition of the patient being treated.
- > Test the patient for tolerability to the tape material.
- > Remove any body hair at the site of use (optional). Please use a long-hair trimmer for this (do not use a wet razor!).
- > Apply the cross tape to clean, dry skin.
- > Make sure that the patient's posture is correct when applying the tape.
- > Make sure that the tape does not arch or form wrinkles.
- > Many tapes are stretched during application, but ensure that measured force is used. The tissue (circulation) must not be constricted.

Combination with other products

No specifications

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling.

Technical data

Sizes:	2.8 × 2.1 cm 3.5 × 2.5 cm 5.2 × 4.5 cm
Colour:	Beige
Material:	Nylon mesh, acrylic adhesive

Maintenance

The product does not require maintenance.

Cleaning instructions

No cleaning instructions.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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ALBO-Healthcare GmbH
TS Co., Ltd.