# Air Pump Walker®

## Air Pump Walker® Achill

### Air Pump Walker®

EN Lower Leg Foot Orthosis  
DE Unterschenkel-Fußorthese  
ES Ortesis de pierna y pie  
FR Orthèse de jambe-pied  
IT Ortesi per piede-gamba  
NL Onderbeen-voetorthese  
DA Underbens-fodortose  
SV Underben-fotortos  
PL Orteza podudzia i stopy  
SL Ortoza meča-stopalo  
HR Ortoza za goljenicu i stopalo  
PT Órtese de pé e perna inferior

### Air Pump Walker® Achill

EN Lower Leg Foot Orthosis with Achilles Tendon Relief  
DE Unterschenkel-Fußorthese mit Achillessehnenentlastung  
ES Ortesis de pierna y pie con descarga del tendón de Aquiles  
FR Orthèse de jambe-pied avec décharge du tendon d'Achille  
IT Ortesi per piede-gamba inferiore con sgravio del tendine di Achille  
NL Onderbeen-voetorthese met achillespeesontlasting  
DA Underbens-fodortose med akillesene-aflastning  
SV Underben-fotortos med hälseneavlastning  
PL Orteza podudzia i stopy z odciążeniem ścięgna Achillesa  
SL Ortoza meča-stopalo z razbremenitvijo Ahilove tetive  
HR Ortoza za goljenicu i stopalo s rasterećenjem Ahilove tetive  
PT Órtese de pé e perna inferior com alívio do tendão de Aquiles

# DARCO

## Air Pump Walker®

## Air Pump Walker® Achill

**EC REP**

EN EU Authorised Representative  
 DE Bevollmächtigter in der EU  
 ES Representante autorizado en la UE  
 FR Représentant autorisé UE  
 IT Rappresentante Autorizzato nel EU  
 NL Wettelijke vertegenwoordiger EU  
 DA EU-autoriseret repræsentant  
 SV Auktoriserad representant i EU  
 PL Autoryzowany przedstawiciel w UE  
 SL Pooblaščen zastopnik EU  
 HR Ovlašteni zastupnik u EU  
 PT Representante autorizado na UE

**UK REP**

EN Responsible Person in the UK  
 DE Verantwortliche Person im Vereinigten Königreich  
 ES Persona responsable en el Reino Unido  
 FR Personne responsable Royaume-Uni  
 IT Rappresentante Autorizzato nel Regno Unito  
 NL Verantwoordelijke rechtspersoon VK  
 DA Befuldmægtiget i Det Forenede Kongerige  
 SV Auktoriserad representant i Storbritannien  
 PL Osoba odpowiedzialna w Wielkiej Brytanii  
 SL Odgovorna oseba v Združenem kraljestvu  
 HR Odgovorna osoba u UK  
 PT Responsável no Reino Unido

**CH REP**

EN Authorised Swiss representative  
 DE Bevollmächtigter in der Schweiz  
 ES Representante autorizado en Suiza  
 FR Mandataire en Suisse  
 IT Rappresentante Autorizzato Svizzera  
 NL Gevolmachtigde in Zwitserland  
 DA Befuldmægtiget i Schweiz  
 SV Anmält organ i Schweiz  
 PL Autoryzowany przedstawiciel w Szwajcarii  
 SL Pooblaščen zastopnik v Švici  
 HR Opunomoćenik u Švicarskoj  
 PT Representante autorizado na Suíça

EN Translation  
 DE Übersetzung  
 ES Traducción  
 FR Traduction  
 IT Traduzione  
 NL Vertaling  
 DA Oversættelse  
 SV Översättning  
 PL Tłumaczenie  
 SL Prevajanje  
 HR Prijevod  
 PT Tradução

EN Observe the instructions for use  
 DE Gebrauchsanweisung beachten  
 ES Observar las instrucciones de uso  
 FR Tenir compte du mode d'emploi  
 IT Consultare le istruzioni per l'uso  
 NL Raadpleeg de gebruiksaanwijzing  
 DA Følg brugsanvisningen  
 SV Läs bruksanvisningen  
 PL Przestrzegaj instrukcji użycia  
 SL Upoštevajte navodila za uporabo  
 HR Pridržavati se uputa za uporabu  
 PT Consultar as instruções de utilização

EN Certification mark declaring this product's compliance with EU legislation  
 DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der EU-Gesetzgebung bestätigt  
 ES Marca de certificación que declara la conformidad de este producto con la legislación de la UE  
 FR Marque de certification attestant de la conformité de ce produit à la législation de l'UE  
 IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione UE  
 NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de EU  
 DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med EU-lovgivningen  
 SV Märket deklarerar att produkten uppfyller lagarna i EU  
 PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami UE  
 SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo EU  
 HR Certifikacijska oznaka koja označava sukladnost ovog proizvoda sa zakonodavstvom EU  
 PT Marca de certificação que declara a conformidade deste produto com a legislação da UE

EN Certification mark declaring this product's compliance with UK legislation  
 DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der Gesetzgebung des Vereinigten Königreiches bestätigt  
 ES Marca de certificación que declara la conformidad de este producto con la legislación de GB  
 FR Marque de certification attestant de la conformité de ce produit à la législation de l'RU  
 IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione del Regno Unito  
 NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de VK  
 DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med UK-lovgivningen  
 SV Märket deklarerar att produkten uppfyller lagarna i Storbritannien  
 PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami Wielkiej Brytanii  
 SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo Združenega kraljestva  
 HR Certifikacijska oznaka koja označava sukladnost ovog proizvoda sa zakonodavstvom UK  
 PT Marca de certificação que declara a conformidade deste produto com a legislação do Reino Unido

EN Date of manufacture  
 DE Herstellungsdatum  
 ES Fecha de fabricación  
 FR Date de fabrication  
 IT Data di fabbricazione  
 NL Productiedatum  
 DA Fremstillingsdato  
 SV Tillverkningsdatum  
 PL Data produkcji  
 SL Datum izdelave  
 HR Datum proizvodnje  
 PT Data de fabrico

## EN Instructions for use

**Air Pump Walker®** | Lower Leg Foot Orthosis

**Air Pump Walker® Achill** | Lower Leg Foot Orthosis with Achilles Tendon Relief

### Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for

direct skin contact. You should use bandages, dressings or socks. We recommend that you do not drive a vehicle while you are wearing this aid.

Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This device is intended for single-patient use only. There is a risk of thrombosis due to the restricted ankle movement. If there are vascular disorders in the body region treated, discuss use with your doctor.

There is a risk that the orthosis may break if the load is too heavy. Max. permitted load: 200 kg. The air chambers must not be over inflated to prevent pressure points on the leg and to prevent the air chambers from bursting.

## Purpose

**Air Pump Walker®:**

Lower Leg Foot Orthosis for immobilisation in a given position

**Air Pump Walker® Achill:**

This medical device is a lower leg foot orthosis for complete immobilisation of the lower leg and foot region in a defined, adjustable position

## Indications / Contraindications

### Indications

**Air Pump Walker® / High shaft:**

- > Pre- and postoperative / post-traumatic immobilisation in case of damage to the bone / capsule and ligament structures of the foot / ankle
- > Soft tissue injury
- > Stable foot and / or ankle fractures
- > Severe sprains grade II and III

**Air Pump Walker® / Short shaft:**

- > Pre- and postoperative / post-traumatic immobilisation in case of damage to the capsule and ligament structures of the foot / ankle
- > Soft tissue injury
- > Stable foot fractures
- > Sprains grade II

**Air Pump Walker® Achill / High shaft:**

- > Achilles tendon ruptures (due to using Achilles Wedges)

### Contraindications

- > Unstable fractures
- > Skin diseases in the body part treated
- > Sensitivity disorders

## Design features

The Lower Leg Foot Orthosis is available in five sizes with a universal fit for right and left. The stable frame construction with perforations for air circulation achieves specific immobilisation of the lower leg / foot with air cushioning while mobilising the patient.

## Maintenance

The product does not require maintenance.

1. Open the circular hook and loop fasteners and remove the front splint.
2. Put on the stocking included in the device packaging.
3. Open the blue liner and place the leg in the liner.
4. Close the liner all the way around. Ensure a wrinkle-free fit.
5. Position the front splint centrally on the blue liner. Ensure that the black pump is centred in the recess.
6. Guide the hook and loop straps through the loops. Pull the hook and loop straps as tight as necessary. Ensure that the leg is supported comfortably.
7. To achieve additional stabilisation of the ankle, the volume of the air chambers can be increased using the pump attached to the side.  
Note: The valve wheel must be turned in the "INFLATE" direction.
8. To reduce the volume, turn the valve wheel to the "DEFLATE" position.  
This allows for individual filling and adjustment.

**Toe Protector:** For apical lesions or wounds on the toes, the toe protector can be removed or relocated.

**Note**

If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers using the valve. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

**Use of the Achilles version:**

9. Open the hook and loop straps, the front splint and the lining.
10. Place the wedges in the heel area of the splint on the lining, corresponding to the degree of plantar flexion that is therapeutically required. A wedge has a height of approx. 10 mm. Depending on the number of wedges used, the height varies and the angle of plantar flexion can, thus, be adjusted individually.
11. Fix the wedges using the adhesive dots.
12. Depending on the course of therapy, the wedges can be removed and, thus, the height adjusted.
13. Once the wedges are positioned, carry out steps 2 to 7.

## Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling/cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

## Technical data

Sizes	Universal fit for right and left	Size	Rigid inner sole length	EU shoe sizes
Colour	Grey-Blue			
Material	Frame: PP	XS	24 cm	34–36
	Outsole: TPR	S	26 cm	37–39
	Liner: PES, PU	M	28 cm	40–42
	Wedges: EVA	L	30 cm	43–45
	Stocking: Polyester, Cotton	XL	32 cm	46–48

## Combination with other products

**Air Pump Walker® / High shaft:** APWAW (Air Pump Walker® Achilles Wedges)

## Care instruction

- > **Shell:** Clean it with a damp cloth or soft brush.
- > **Lining/Stocking:** Hand wash in cold water (30° C). Using mild soap. Do not bleach.
- > Let the shell or liner dry in the air only. Do not dry the components on direct heat or sun.

**Note**

Pump can not be removed from the lining.

## Disposal

Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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CMC medical devices Ltd.
CMC medical devices GmbH
Span Link International, LLC