# Rain Cover

- DE Regenschutz
- EN Rain protection
- ES Protección contra la lluvia
- FR Couvre-chaussure
- IT Copertura antipioggia
- NL Bescherming tegen regen
- DA Regnbeskyttelse
- SV Regnskydd
- PL Osłona przed deszczem
- SL Zaščita pred dežjem
- HR Zaštita od kiše
- PT Resguardo para proteção contra chuva

# DARCO

## Rain Cover | Regenschutz

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## Rain Cover | Rain protection

### Important notes

Please always follow these instructions for use. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This product is not recommended for direct skin contact.

You must not drive a vehicle while you are wearing this product. Use extreme caution when climbing stairs (e. g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

## Purpose

The rain protection protects the foot from dirt, moisture and rain, and therefore improves wearing comfort.

## Design features

The water-repellent material protects the foot. The non-slip tread of the shoe is not covered by the rain protection.

## Use

1. Put on the lower leg foot orthosis or shoe as instructed.
2. Open the zip of the rain protection
3. Position the toe of the shoe in the rain protection cover (Fig. 1)
4. Pull the elastic band over the heel (Fig. 2)
5. Close the zip (Fig. 3)
6. Pull the rubber cord tightly at the top end (Fig. 4)

## Combination with other products

Can be combined with DARCO shoes and orthoses, e. g. Air Pump Walker® / Achill, Relief Insert® Walker 2.0 / Achill MedSurg / Pro®, Relief Dual® / Plus, SlimLine®

## Working life / Lifespan of the product

The lifespan of the product is determined by natural wear and tear due to proper handling / cleaning.

## Technical data

**Size 1:** For the products XS, S, M and WS, WM, WL, MS, MM  
Universal fit for right and left

**Size 2:** For the products L, XL and ML, MXL  
Universal fit for right and left

**Colour:** Black

**Material:** Polyester and PVC

## Care instructions

- > If the product is slightly soiled, it can be wiped with a damp cloth.
- > Do not dry the product in a dryer or on a radiator. Leave it to dry at room temperature.

## Maintenance

The product does not require maintenance.

## Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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