MedSurg Pediatric

EN Post-Op Shoe for Children

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EN Instructions for use

MedSurg Pediatric | Post-Op Shoe for Children

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO International Inc. does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks.

You must not drive a vehicle while you are wearing this product. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

Note

This device is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

The MedSurg Post-Op Shoe provides stability and protects the foot against bumps and reduces pressure on it following foot surgery.

Indications / Contraindications

Indications

Postoperative for stabilisation and offloading, e.g. foot osteotomies

Contraindications

Open fractures

Design features

The upper material is made of breathable polyester material. The sole is made from EVA with shock absorbing effect. The sole profile facilitates the rolling off movement. Universal fit for right and left.

Combination with other products

No information

1. Open the shoe using the hook and loop straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The inner shoe tabs can optionally be placed over each other.
4. The shoe can be visually adapted to the left or right foot by the positioning of the removable hook and loop strap (B). The logo must be located on the outside of the foot.
5. Fasten the shoe using the hook and loop straps. If needed, the length of the upper hook and loop strap (A) can be customised to fit.

Technical data

Sizes	1 child size: 27–32
Colour	Black
Material	Outsole: EVA, bamboo Insole: EVA Upper material: PU, polyester Liner: Polyester Hook-and-loop fastener: Nylon

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Maintenance

The product does not require maintenance.

Care instructions

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Leave the shoe to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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DARCO International, Inc.
DARCO (Europe) GmbH
V-M Orthotics Ltd.
ALBO-Healthcare GmbH