# Body Armor<sup>®</sup> Night Splint

## + Toe Plate (Zehenplatte)

- EN Dorsal Foot-Positioning Orthosis + Toe Plate  
DE Dorsale Fußlagerungsorthese + Zehenplatte  
ES Ortesis de posicionamiento dorsal del pie + Placa para dedos del pie  
FR Orthèse de positionnement dorsal du pied + plaque pour orteil  
IT Ortesi per il posizionamento dorsale del piede + Piastra salvadita  
NL Dorsale voetimmobilisatie-orthese + teenplaat  
DA Dorsal fodlejringsortose + tåplade  
SV Dorsal fotavlastningsortos + tåplatta  
PL Grzbietowa orteza pozycjonująca stopę + płytką ochronną na palce nóg  
SL Dorzalna ortoza za imobilizacijo stopala + ploščica za nožne prste  
HR Dorzalna ortoza za impobilizaciju stopala + pločica za nožne prste  
PT Órtese de posicionamento do pé dorsal + Placa do dedo grande do pé

# DARCO

### Body Armor® Night Splint | Dorsal Foot-Positioning Orthosis + Toe Plate

#### Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO International Inc. does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks. The orthosis is not designed for walking. You must not drive a vehicle while you are wearing this product.

**Note:** This device is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

## Purpose

The Body Armor® Night Splint is a backing splint used for the treatment of heel spurs, plantar fasciitis, shortening of the plantar fascia, talipes equinus or achilles tendonitis.

## Indications / Contraindications

#### Indication

- > Shortening of the plantar fascia
- > Plantar fasciitis / heel spur
- > Achilles tendonitis and weaknesses
- > Metatarsalgia
- > talipes equinus and ankle contracture

#### Contraindication

- > Plantar flexion contracture greater than 10°
- > uncontrollable severe plantar flexion spasticity
- > This product is not intended for extended periods of weight bearing or ambulation.
- > In case of an extremely upright Hallux Valgus position we recommend to use the Toe plate instead of the toe loop.

## Design features

The Body Armor® Night Splint backing splint maintains the foot and lower leg at a 90° position. The elastic ring connector allows for off-loading the instep of the foot. The loops are made of breathable, elastic nylon / polyester fabric.

## Combination with other products

#### Toe plate:

The Toe plate is an addition to the Body Armor® Night Splint splinting system. It provides the elevation of the five toes. This enables an almost complete even stretching of the flexor group.

## Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

## EN Use

1. Slide foot into the night splint, seating the heel snugly in the heel opening.
2. Tighten, as needed, both sides of the heel strap together.
3. Tighten, as needed, both sides of the foot strap together.
4. Thread the calf strap. Tighten, as needed, both sides of the calf strap together.

### Application of the Toe loop:

5. Insert the big toe in the toe loop. The top of the toe should be level with the end of the toe loop.
6. Using the elastic ring, place the big toe in a neutral or slightly elevated position\* and attach the ring to one of the hooks.
7. Secure the toe loop tab to the bottom of the night splint.

### Application of the Toe plate:

8. Insert the 2. and/or 3. toe into the toe loop. The top of the toe should be level with the edge of the toe plate.
9. Using the elastic ring, place the big toe in a neutral or slightly elevated position\* and attach the ring to one of the hooks.
10. Ready with the application of the toe plate.

\* As treatment continues, the toe(s) may be elevated as comfort allows to permit a greater stretch to the plantar fascia. If you experienced any numbness or discomfort in the big toe(s) area, reduce the tension on the toe strap / plate in order to get an optimal and neutral position.

### Important notes:

This product is allowed for single-patient use only. The Body Armor® Night Splint is a backing splint and not intended for walking.

## Technical data

### Size:

Small (34 – 36)  
Standard (37 – 47)  
Universal fit for right and left

### Colour:

White-Blue

### Material:

Shell backing splint: Nylon  
Soft pads: Polyester Spandex  
Elastic ring: PVC  
Hook and loop fastener straps: Nylon  
Toe loop: Polyester Spandex  
Toe plate: Polyester Spandex

## Maintenance

The product does not require maintenance.

## Care instruction

- > Open all the hook and loop fastener straps.
- > The shell and padding may be cleaned by wiping with a damp cloth.
- > The Toe loop / Toe plate is to be washed by hand with a mild detergent. The grey fabric for fixing the splint can be washed with 30°. This product is to be air dried and may not be tumbled dry or dried on the heater.
- > For cleaning the product you may use any commercially available sanitizer.

## Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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DARCO International, Inc.
V-M Orthotics Ltd.
DARCO (Europe) GmbH
ALBO-Healthcare GmbH