Relief Contour Insole

- EN** Arch Cushion Insole
- DE** Weichbettungseinlegesohle
- ES** Plantilla con almohadillado blando
- FR** Semelle voûte plantaire coussinée
- IT** Soletta morbida
- NL** Zachte inlegzool
- DA** Blød indlægssål
- SV** Mjuk inläggssula
- PL** Miękka wkładka amortyzująca
- SL** Mehek notranji podplat
- HR** Uložak s mekanom podlogom
- PT** Palmilha de amortecimento suave com apoio em arco

Important notes

 If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, they will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly.

This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks. Do not drive a vehicle while you are wearing this device in a special shoe.

Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

 **Note:** This device is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

Cushion Insole for redistribution of pressure

Indications / Contraindications

Indications

- For use following hallux valgus and small toe surgeries (Austin, Akin, Scarf, Weil, Hohmann, etc.)
- Talipes valgus, flat foot, splayfoot
- Angioneuropathic changes of the foot in combination with mild foot deformities

Contraindications

- Open fractures
- Not suitable for the treatments of the hind foot (negative heel)
- Marked foot deformities of the diabetic foot

Design Features

The Relief Contour Insole is constructed with a negative heel of 2.4 degrees in dorsal extension. This results in mild hind foot pressure and simultaneous forefoot off-loading. The transverse arch as well as medial and lateral longitudinal arch offer perfect pressure redistribution. Pressure peaks are reduced by the long-sole, continuous soft foot bed.

Use

1. Remove the original insole from the shoe.
2. Insert the right or left Relief Contour Insole in the shoe in place of the original insole.

 **Note:** Using the insole creates a right or left shoe from a shoe that can be worn on both sides. If needed, the PU material can be trimmed, which enables individual, patient-specific adjustment of the Relief Contour Insole.

Combination with other products

- Relief Dual® / Plus
- Relief Insert® Walker 2.0
- MedSurg Pro®

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

Sizes: Right and left version available

Three ladies sizes:

EU WS (34 – 36.5), WM (37 – 38.5), WL (39 – 41)

UK WS (2 – 4), WM (4.5 – 5.5), WL (6 – 7.5)

Four mens sizes:

EU MS (39 – 41), MM (41.5 – 43), ML (43.5 – 45), MXL (45.5 – 47)

UK MS (5.5 – 8), MM (8.5 – 10), ML (10.5 – 12), MXL (12.5 – 14)

Colour: Blue

Material: The insole is made of polyurethane. The insole covering is made of polyester.

Maintenance

The product does not require maintenance.

Care instructions

The insole can be cleaned with lukewarm water or with conventional disinfectants. Do not dry the insole on a heating unit; instead allow it to dry at room temperature.

Disposal

 Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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DARCO UK Ltd
ALBO-Healthcare GmbH