Relief Insert® Walker 2.0

Relief Insert® Walker 2.0 Achill*

- EN** Lower Leg Foot Orthosis / * with Achilles Tendon Relief
- DE** Unterschenkel-Fußorthese / * mit Achillessehnenentlastung
- ES** Ortesis de pierna y pie / * con descarga del tendón de Aquiles
- FR** Orthèse de jambe-pied / * avec décharge du tendon d'Achille
- IT** Ortesi per piede-gamba inferiore / * con sgravio del tendine di Achille
- NL** Onderbeen-voetorthese / * met achillespeesontlasting
- DA** Underbens-fodortose / * med akillesene-aflastning
- SV** Underben-fotortos / * med hälseneavlastning
- PL** Orteza podudzia i stopy / * z odciążeniem ścięgna Achillesa
- SL** Ortoza meča-stopalo / * z razbremenitvijo Ahilove tetive
- HR** Ortoza za goljenicu i stopalo / * s rasterećenjem Ahilove tetive
- PT** Órtese de pé e perna inferior / * com alívio do tendão de Aquiles

DARCO

Relief Dual® Plus

PegContour® Insole Uni for Relief Insert® Walker 2.0

Relief Insert® 2.0

E

- EN Strap loops
 DE Umlenkschlaufen
 ES Lazos de desviación
 FR Boucles de sangles
 IT Passanti
 NL Doorhaallus
 DA Stropper
 SV Remögla
 PL Klamry do przewlekania
 SL Zanke
 HR Skretne omče
 PT Fivelas

F

- EN Forefoot strap
 DE Vorfußgurt
 ES Correa del antepié
 FR Sangle d'avant-pied
 IT Cinturino sull'avampiede
 NL Voorvoetband
 DA Forfodsrem
 SV Framfotsrem
 PL Pasek na przedniej części stopy
 SL Trak za sprednji del stopala
 HR Pojas za prednji dio stopala
 PT Correia de antepé

G

- EN Pump to fill the air liner air chambers
 DE Pumpe zum Befüllen der Luftkammern des Air-Liners
 ES Bomba para inflar las cámaras de aire del forro inflable
 FR Pompe de gonflage des alvéoles de l'Air-Liner
 IT Pompa per il riempimento delle camere d'aria dell'Air-Liner
 NL Pomp voor het vullen van de luchtkamers van de air-liner
 DA Pumpe til at fylde luftkamrene i Air-lineren
 SV Pump för påfyllning av luftkamrarna i Air-Liner
 PL Pompka do napełniania komór powietrznych wkładki powietrznej
 SL Črpalka za polnjenje zračnih komor na vložku Air-Liner
 HR Pumpa za punjenje zračnih komora zračne podstave
 PT Bomba para encher as câmaras de ar do revestimento Air

H

- EN Valve for releasing the air from the air chambers
 DE Ventil zum Ablassen der Luft aus den Luftkammern
 ES Válvula para desinflar las cámaras de aire
 FR Valve d'évacuation de l'air des alvéoles
 IT Valvola per lo scarico dell'aria dalle camere d'aria
 NL Ventiel waardoor lucht uit de luchtkamers kan ontsnappen
 DA Ventil til udledning af luft fra luftkamrene
 SV Ventil för att släppa ut luft ur luftkamrarna
 PL Zaworek do spuszczenia powietrza z klamer powietrznych
 SL Ventil za izpuščanje zraka iz zračnih komor
 HR Ventil za ispuštanje zraka iz zračnih komora
 PT Válvula de saída do ar das câmaras de ar

I

- EN Calf splint with integrated air liner with air chambers
- DE Waden-Schiene mit fest integrier-ten Air-Liner mit Luftkammern
- ES Férula para pantorrilla con forro inflable integrado fijo y cámaras de aire
- FR Attelle de mollet avec Air-Liner intégré et alvéoles
- IT Stecca per polpaccio con Air-Liner integrato fisso e dotato di camere d'aria
- NL Kuitbrace met vast geïntegreerde air-liner met luchtkamers
- DA Læg-skinne med fast integreret Air-liner med luftkamre
- SV Vadskenamed fast Air-Liner med luftkammare
- PL Szyna tydkowa z umocowaną na stałe wkładką powietrzną z komo-rami powietrznymi
- SL Opornica za meča s fiksno integri-ranim vložkom Air-Liner z zračnimi komorami
- HR Udlaga za list s čvrsto ugrađenom zračnom podstavom sa zračnim komorama
- PT Tala dos gémeos com revestimen-to Air integrado de forma fixa com câmaras de ar

J

- EN Automatic modification of the width: For unhooking, if necessary
- DE Automatische Weitenregulierung: Im Bedarfsfall zum Aushaken
- ES Regulación automática de la anchura: Se puede desenganchar en caso necesario
- FR Réglage automatique de la largeur : Si nécessaire pour décrocher
- IT Regolazione automatica della larghezza: Se necessario per sganciare
- NL Automatische wijtdeaanpassing: Waar nodig los te haken
- DA Automatisk bredderegulering: Kan hægtes af efter behov
- SV Automatisk viddreglering: Kan hakas loss vid behov
- PL Automatyczna regulacja szerokości: W razie potrzeby można rozpiąć
- SL Samodejna regulacija širine: Za odpenjanje po potrebi
- HR Automatska regulacija širine: Može se otkopčati po potrebi
- PT Regulação automática em largura: Se necessário, para desenganchar

K

- EN Calf liner
- DE Waden-Liner
- ES Forro para pantorrilla
- FR Doublure du mollet
- IT Rivestimento del polpaccio
- NL Kuitvoering
- DA Læg-liner
- SV Vadliner
- PL Wkładka na łydce
- SL Vložek za meča
- HR Podstava za list
- PT Revestimento dos gêmeos

L

- EN Air liner with air chambers
- DE Air-Liner mit Luftkammern
- ES Forro inflable con cámaras de aire
- FR Air-Liner avec alvéoles
- IT Air-Liner con camere d'aria
- NL Air-liner met luchtkamers
- DA Air-Liner med luftkamre
- SV Air-Liner med luftkammare
- PL Wkładka powietrzna z komorami
- SL Air-Liner z zračnimi komorami
- HR Zračna podstava sa zračnim komorama
- PT Revestimento Air com câmaras de ar

M

- EN Front splint with front liner
- DE Front-Schiene mit Front-Liner
- ES Férula frontal con forro frontal
- FR Attelle avant avec doublure avant
- IT Stecca anteriore con rivestimento anteriore
- NL Frontbrace met frontvoering
- DA Front-skinne med Front-liner
- SV Frontskena med frontliner
- PL Szyna frontowa z wkładką frontową
- SL Sprednja opornica s sprednjim vložkom
- HR Prednja udloga s prednjom podstavom
- PT Tala dianteira com revestimento dianteiro

Relief Insert® Walker 2.0 | Lower Leg Foot Orthosis

Relief Insert® Walker 2.0 Achill | Lower Leg Foot Orthosis with Achilles Tendon Relief

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. You must not drive a vehicle while you are wearing this product.

Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

Note

This aid is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses. There is a risk of thrombosis due to the restricted ankle movement. There is a risk that the orthosis (splint) may break if the load is too heavy. Max. permitted load: 150 kg. The air chambers must not be over inflated to prevent pressure points on the leg and to prevent the air chambers from bursting. The air liner with pump cannot be washed.

Purpose

Relief Insert® Walker 2.0: Lower leg / foot orthosis for immobilisation in a given position, as well as a foot-positioning orthosis

Relief Insert® Walker 2.0 Achill: Lower leg / foot orthosis and foot-positioning orthosis with Achilles tendon relief

Indications / Contraindications

Relief Insert® Walker 2.0

Indications

- Arthrodesis of the first TMT joint (lapidus arthrodesis)
- After metatarsal fracture surgery
- After hallux valgus surgery
- After hallux rigidus surgery
- After hammer and claw toe surgery
- Fractures of the phalanges
- After surgery on the rearfoot
- For ligament rupture of the ankle joint
- Postoperative use after removal of Haglund's exostosis (with intact Achilles tendon)
- Pre- and postoperative / post-traumatic immobilisation of the lower leg / foot, stable foot or ankle fractures, severe ankle sprains
- Upper ankle joint arthrodesis
- Postoperatively after Weber fractures

Contraindications

- Acute Weber fractures (B, C)
- Unstable fractures

Relief Insert® Walker 2.0 Achill

Indications

- Achilles tendon ruptures
- Postoperative use for Achilles tendon suture
- Postoperative use after removal of Haglund's exostosis

Contraindications

- Forefoot conditions
- Plantar foot ulcers

Design features

The lower leg / foot orthosis is available in five sizes with a universal fit for right and left. The shoe can be customized to the left or right foot by individually adjusting the hook and loop straps. The lower leg / foot orthosis consists of the following components: One shoe (Relief Dual® Plus | Off-loading shoe with high ankle support), a splint (Relief Insert® 2.0 | Splinting System) with a grey air liner and integrated pump and a blue lining (consists of a forefoot strap, front and calf liner). The Relief Insert® Walker 2.0 Achill also includes heel wedges (DAW | DARCO Achilles Wedges).

The splinting system is used to position the lower leg / foot. In addition, a walker can be assembled from the splinting system using a therapeutic shoe to immobilise the lower leg / foot with simultaneous mobilisation. The exchangeable insole enables patients to individually adjust the fit.

Use

To avoid the hook and loop straps getting in the way during the individual application steps, attach them behind onto the respective fleece side.

Using the Relief Insert® 2.0 as a splinting system:

Using the Relief Insert® 2.0 as a splinting system:

When used as a splinting system, the Relief Dual® Plus Off-loading Shoe is not required. Set the shoe aside and proceed with opening the splinting system.

1. Fold the front splint away to the side.
2. If desired, another compatible DARCO insole can be placed in the splinting system. To do this, remove the blue insole. Then place and fix the desired insole analogously to the blue insole in the splinting system.
3. Place the leg on the calf liner of the calf splint. Make sure that the foot is in contact with the insole.
4. The width of the guide splint automatically adjusts to the patient's needs by means of the adjuster on the back of the guide splint.
5. Once the leg is positioned, close the splinting system. To do this, place the front splint on the shin. Make sure that the front splint is positioned centrally within the calf splint. Guide the three hook and loop straps through the loops. First close the lowest strap at the ankle and then the two upper hook and loop straps. Pull the hook and loop straps as tight as necessary. Ensure that the leg is supported comfortably. Optionally, the blue forefoot strap (F) can be attached to the forefoot for additional support.

Using the Relief Insert® Walker 2.0 as a lower leg / foot orthosis:

Perform steps 1 and 2 as described. After that, please continue as follows.

- 2a. Remove the insole from the splint.
- 2b. Determine the area of the foot that needs relief.
- 2c. Carefully remove the pegs at the site at which the pressure should be relieved.
Use a suitable tool for this or remove the pegs by hand.
- 2d. Reinsert the adapted PegContour® Insole Uni in the shoe using the hook and loop fastener with the pegs facing downwards.

Note: Signs of fatigue can occur in the cover layer when there is a large gap (over 3 cm diameter).

Perform steps 3 – 5 as described. After that, please continue as follows.

6. Let the patient step into the opened foot section of the Relief Dual® Plus Off-loading Shoe with the orthosis (splint) on. For particularly easy entry, open the Relief Dual® Plus completely using the hook and loop straps (A, B, C). Make sure that the insole is positioned securely at the front in the shoe.
7. Now close the off-loading shoe with the two hook and loop straps C of the heel cap.
8. The inner shoe tabs can be placed over each other either way (D).
9. Close the two hook and loop straps A, B as described below. Fold the strap A on the outer side (left or right) around the tab and secure it firmly. Close the shoe with the strap on the other side. If needed, the length of the upper hook and loop strap A can be customised to fit. Now close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.
10. To achieve additional stabilisation of the ankle joint, the volume of the air chambers can be increased using the pump attached to the side. To reduce the volume, release the air using the silver valve. This allows for individual filling and adjustment. Ensure that the leg is supported comfortably. If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers using the valve. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

Using the Relief Insert® Walker 2.0 Achill as an Achilles walker:

11. Fold the front splint to the side.
12. Leave the insole in the splint.
13. Place the wedges in the heel area of the splint on the grey insole, corresponding to the degree of plantar flexion that is therapeutically required. A wedge has a height of approx. 11 – 12 mm. Depending on the number of wedges used, the height varies and the angle of plantar flexion can, thus, be adjusted individually.
3 wedges are equivalent to approx. 120° slant.
14. Fix the wedges to the insole using the adhesive dots.

15. Depending on the therapy course, the wedges can be removed and, thus, the height adjusted.

Once the wedges are positioned, please carry out steps 3 to 10.

Changing the lining

Removing the lining:

16. Remove the blue calf liner attached to the calf splint with hook and loops.
17. The grey air liner with integrated air chambers and pump remains in the product and must not be removed.
18. Now carefully remove the blue front liner attached to the front splint with hook and loop fasteners.
19. The calf liner, front liner and forefoot strap can be washed, if required.

Fitting the lining:

After cleaning the blue lining, please carry out the steps described below:

20. First attach the front liner to the front splint using the hook and loop fasteners.
To do this, fold the liner in the middle lengthwise and place it centrally (according to the shape of the front splint) on the front splint. By unfolding the liner, it clings to the front splint in the correct position. Gently press the front liner against the splint.
21. The calf liner is also folded lengthwise in the middle for attachment to the splint.
Place the calf liner in the centre of the calf splint with the notches at the level of the ankle. By unfolding the liner, it clings to the calf splint in the correct position.
Gently press the calf liner.

To close the splinting system or walker, follow the steps described from step 1.

Important notes

The Relief Insert® Splinting System may only be used in combination with the Relief Dual® Plus Off-loading Shoe for walking. The splint is not designed for walking without a shoe. It may only be used for positioning.

The blue PegContour® Insole Uni inserted in the Relief Insert® Walker 2.0 is specially designed for the Relief Insert® Walker 2.0 and is not compatible with other products.

Combination with other products

- PegAssist® Insole
- Puzzle Insole®
- PegContour® Insole Uni (blue)
- Relief Contour Insole
- Toe Cap
- DAW (DARCO Achilles Wedges)
- Relief Insert® 2.0 Lining
- Rain Cover

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

Sizes

XS 37.0 – 38.5

S 39.0 – 41.0

M 41.5 – 43.0

L 43.5 – 45.0

XL 45.5 – 47.0

Colour

Grey-Blue

Material

Outsole: Rubber, EVA

Insole grey: EVA

Insole blue: Polyurethan, polyester, nylon

Upper material of shoe: Polyester foam, microfibre

Lining and forefoot strap: Polyester

Hook and loop straps: Nylon

Adhesive: Polyurethane glue

Splints: Polyamide

Pump: TPU

DAW (heel wedges): EVA

Maintenance

The product does not require maintenance.

Care instructions

Washing

Shoe, splint, insole, wedges: If these are slightly soiled, wipe with a damp cloth or clean with commercially available disinfectants.

Liners: To wash, the blue calf and front liners are removed from the splint and then washed at approx. 60° C in a washing machine. The optional forefoot strap can also be washed at 60° C. The air liner with air chambers and the pump cannot be washed.

Drying

Do not dry any of the components of the Relief Insert® Walker 2.0 | Achill in a dryer or on a radiator, but dry them at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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ALBO-Healthcare GmbH
DARCO UK Ltd