MedSurg Pro[®]

- EN** Healing Shoe in low and high versions
- DE** Verbandschuh in niedriger und hoher Version
- ES** Zapato postquirúrgico en versión alta y baja
- FR** Chaussure postopératoire en version basse et haute
- IT** Calzatura post-operatoria nella variante bassa e alta
- NL** Postoperatieve schoen in een lage en hoge versie
- DA** Bandagesko i lav og høj version
- SV** Postoperativ sko i låg och hög version
- PL** But pooperacyjny w wersji niskiej i wysokiej
- SL** Začasni terapevtski čevlji v nizki in visoki različici
- HR** Ortopedska cipela u niskoj i visokoj verziji
- PT** Sapato pós-operatório em versão baixa e alta

DARCO

MedSurg Pro®

MedSurg Pro® | Healing Shoe in low and high versions

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks.

You must not drive a vehicle while you are wearing this product. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

Note: This device is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

The MedSurg Pro® Healing Shoe in low and high versions provides stability and protects the foot against bumps and reduces pressure on it following foot surgery.

Indications / Contraindications

Indications for the low and high versions

- Postoperative for stabilisation and offloading, e.g. foot osteotomies

Contraindications for the low and high versions

No contraindications are known.

Design Features

The shoe upper is made of a breathable fabric that can be stretched lengthways and sideways, and the slip-resistant sole is made of EVA. The sole profile facilitates the rolling off movement. Universal fit for right and left.

Use

1. Open the shoe using the hook and loop straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The shoe can be customized to the right or left foot by individually adjusting the hook and loop straps (A, B).
4. Position the upper hook and loop strap A to the left or right so that the hook strap on both sides is on the outside of the shoe. Fold it around the tab and secure it firmly. If needed, the length of the upper hook and loop strap can be customized to fit.
5. The inner shoe tabs can optionally be placed over each other (C).
6. Close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.

Combination with other products

- Puzzle Insole®
- PegContour® Insole Uni
- PegAssist® Insole
- Relief Contour Insole
- Toe Cap

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

Sizes

3 women's sizes: WS (34 – 36.5), WM (37 – 38.5), WL (39 – 41)

4 men's sizes: MS (39 – 41), MM (41.5 – 43), ML (43.5 – 45), MXL (45.5 – 47)

Colour

Black | Low and high (ankle-spanning) version

Material

Outsole: EVA, bamboo

Insole: EVA

Upper material: PU, polyester

Liner: Polyester

Hook-and-loop fastener: Nylon

Maintenance

The product does not require maintenance.

Care instructions

1. Remove coarse dirt from the outsole and upper material with a dry soft shoe brush.
2. Apply a mild soap solution to a soft brush or cloth. Clean the outsole thoroughly with it. Dry the sole with a soft cloth.
3. Remove the insole and wash it by hand with mild soap or disinfect it. Make sure the insole is completely dry before putting it back in the shoe.
4. The upper material can be cleaned with a damp cloth. Do not rub too hard to avoid damaging the materials.
5. Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Caution: Do not use any aggressive cleaning agents or bleach.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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ALBO-Healthcare GmbH
DARCO UK Ltd