OrthoWedge Pro

- EN** Forefoot Off-Loading Shoe
- DE** Vorfußentlastungsschuh
- ES** Zapato de descarga del antepié
- FR** Chaussure de décharge de l'avant-pied
- IT** Calzatura per lo sgravio dell'avampiede
- NL** Voorvoethulpschoen
- DA** Forfodsaflastningssko
- SV** Avlastningssko för framfoten
- PL** But odciążający przodostopie
- SL** Čevelj za razbremenitev sprednjega dela stopala
- HR** Cipela za rasterećenje prednjeg dijela stopala
- PT** Sapato de alívio do antepé

DARCO

OrthoWedge Pro

OrthoWedge Pro | Forefoot Off-Loading Shoe

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks.

You must not drive a vehicle while you are wearing this product. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

Note: This aid is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

Forefoot Off-Loading Shoe following surgery:
Specifically designed for relief and protection of the forefoot

Indications / Contraindications

Indications

- Postoperative use after surgery to correct hallux valgus or hammer / claw toes
- Healing of forefoot ulcerations

Contraindications

- Open fractures

Offloading in the treatment of a diabetic foot ulcer is only indicated when other necessary offloading measures (e.g., a total contact cast or a comparable orthosis) are definitively not possible.

Design Features

The Forefoot Off-Loading Shoe encloses the heel and tarsus with a soft padded ankle support. The special sole construction provides relief for the forefoot. The heel cap, hook and loop straps and fasteners prevent slipping in the shoe. The forefoot area is protected against inadvertent impact. The insole is removable and can be replaced with a DARCO insole.

Use

1. Open the shoe (A).
2. Step into the open shoe.
3. Please ensure that the foot is securely positioned in the shoe.
4. Close the shoe using the hook-and-loop straps (A).

The foot must not be too tightly or too loosely positioned in the shoe.

Note:

5. When walking, only roll to the inner edges (B). Rolling to the outermost edge (C) can impair the surgical outcome. Excessive weight loading (obese patients) can lead to breakage of the functional shoe sole.
6. In case of longer periods of wear, height compensation (TwinShoe®) on the other foot is recommended.

Combination with other products

- TwinShoe®
- PegAssist® Insole
- PegContour® Insole Uni
- Toe Cap

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

Sizes

5 sizes: XS (33.5 – 37), S (37.5 – 39.5), M (40 – 41.5), L (42 – 44), XL (44.5 – 47)

The shoe fits left or right equally.

Colour

Black

Material

Outsole: EVA, bamboo

Insole: EVA

Upper material: Polyester, PU

Hook and loop straps: Nylon

Maintenance

The product does not require maintenance.

Care instructions

- If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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ALBO-Healthcare GmbH
DARCO UK Ltd