TwinShoe®

- EN** Height Compensation Shoe
- DE** Höhenausgleichsschuh
- ES** Zapato compensador de altura
- FR** Chaussure de compensation de la hauteur
- IT** Calzatura di compensazione dell'altezza
- NL** Hoogtecompensatieschoen
- DA** Udligningssko
- SV** Höjdkompensationssko
- PL** But wyrównujący wysokość
- SL** Čevelj za izravnavo višine
- HR** Cipela za izjednačenje visine
- PT** Sapato compensador da altura

DARCO

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.), please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility.

Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use band-aids, dressings or socks.

Do not drive a vehicle while you are wearing this product. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

Note: This product is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

The TwinShoe® is a height compensating shoe for our proven products. The DARCO TwinShoe® guarantees a safe and physiologically correct gait.

Indications / Contraindications

Indications

- Compensation of leg length difference (approx. 45 mm)
- Avoidance of pelvic tilt

Contraindications

- Open fractures

Design Features

The upper is made of a breathable fabric that can be stretched lengthways and side-ways, and the sole is made of shock-absorbing TPR rubber. The sole profile facilitates the rolling off movement.

Use

1. Open the shoe (A).
2. Step into the open shoe.
3. Make sure that the foot is positioned securely in the shoe and the heel is as far back as possible in the shoe.
4. Close the shoe using the hook-and-loop straps (A). The foot must not be too tightly or too loosely positioned in the shoe.

Combination with other products

Can be used with the following products

- HeelWedge®
- OrthoWedge
- WCS®

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. The length of time this device is worn depends on the indication. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

Sizes

5 sizes

UK XS (up to 4), S (4.5 – 6), M (6.5 – 8), L (8.5 – 10), XL (10.5 – 12)

EU XS (33.5 – 37), S (37.5 – 39.5), M (40 – 41.5), L (42 – 44), XL (44.5 – 47)

Universal fit for right and left

Colour

Black

Material

Outsole: TPR

Insole: EVA

Upper material: Polyester

Maintenance

The product does not require maintenance.

Care instructions

- If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- The TwinShoe® can be hand washed in cold water with a mild soapy solution. Please do not use any aggressive cleaning agents (alcohol etc.).
- Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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ALBO-Healthcare GmbH
DARCO UK Ltd