SlimLine®

SlimLine® Pediatric

- EN** Cast Boot
- DE** Gips- und Verbandschuh
- ES** Zapato para escayola y postquirúrgico
- FR** Chaussure pour plâtre et pansement
- IT** Calzatura ortopedica e coprighesso
- NL** Gips- en verbandschoen
- DA** Gips- og bandagesko
- SV** Gips- och förbandssko
- PL** But na gips i opatrunek
- SL** Mavčni in terapevtski čevelj
- HR** Gips cipela i ortopedska cipela
- PT** Sapato pós-operatório para gesso

DARCO

SlimLine®

SlimLine® Pediatric

SlimLine® | Cast Boot

SlimLine® Pediatric | Pediatric Cast Boot

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.), please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility.

Do not wear the product in direct contact with injured, irritated, damaged or broken skin. You should use bandages, dressings or socks.

Do not drive a vehicle while you are wearing this product. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Be particularly cautious when walking in wet, slippery and snowy conditions. Walk more slowly than usual.

Note: This product is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

The SlimLine® is a flat, versatile cast boot for the protection of all types of plaster casts and heavy compression dressings.

Indications / Contraindications

Indications

- For plaster or cast treatments
- For fractures
- For postoperative care
- Bulky dressings

Contraindications

- Open fractures

Design Features

Adult and child model: The upper is made of a breathable EVA / polyester material. The shock-absorbing sole is made of TPR rubber. The sole profile facilitates the rolling off movement.

Use

1. Open the shoe (A).
2. Step into the open shoe.
3. Make sure that the foot is positioned securely in the shoe and the heel is as far back as possible in the shoe.
4. Close the shoe using the hook-and-loop straps (A). The foot must not be too tightly or too loosely positioned in the shoe.

Optional

The insole is removable / replaceable by specialised medical professionals.

Combination with other products

None available

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

Sizes

5 adult sizes (SlimLine®):

XS (33.5 – 37), S (37.5 – 39.5), M (40 – 41.5), L (42 – 44), XL (44.5 – 47)

3 children's sizes (SlimLine® Pediatric):

S (up to 21), M (22 – 26), L (27 – 32)

Universal fit for right and left.

Colour

Adult model (SlimLine®): Black

Children's model (SlimLine® Pediatric): Blue or pink-blue

Material

Outsole: TPR

Insole: EVA

Upper material: EVA / Polyester

Hook and loop straps: Nylon

Maintenance

The product does not require maintenance.

Care instructions

- Do not machine wash
- If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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DARCO UK Ltd
ALBO-Healthcare GmbH
DARCO International, Inc.