Air Pump Walker®

Air Pump Walker® Achill *

- EN** Lower Leg Foot Orthosis / * with Achilles Tendon Relief
DE Unterschenkel-Fußorthese / * mit Achillessehnenentlastung
ES Ortesis de pierna y pie / * con descarga del tendón de Aquiles
FR Orthèse de jambe-pied / * avec décharge du tendon d'Achille
IT Ortesi per piede-gamba inferiore / * con sgravio del tendine di Achille
NL Onderbeen-voetorthese / * met achillespeesontlasting
DA Underbens-fodortose / * med akillesene-aflastning
SV Underben-fotortos / * med hälseneavlastning
PL Orteza na podudzie i stopę / * z odciążeniem ścięgna Achillesa
SL Ortoza meča-stopalo / * z razbremenitvijo Ahilove tetive
HR Ortoza za goljenicu i stopalo / * s rasterećenjem Ahilove tetive
PT Órtese de pé e perna inferior / * com alívio do tendão de Aquiles

DARCO

Air Pump Walker®

Air Pump Walker® Achill

Air Pump Walker® | Lower Leg Foot Orthosis

Air Pump Walker® Achill | Lower Leg Foot Orthosis with Achilles Tendon Relief

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks. We recommend that you do not drive a vehicle while you are wearing this aid.

Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This device is intended for single-patient use only. There is a risk of thrombosis due to the restricted ankle movement. If there are vascular disorders in the body region treated, discuss use with your doctor.

There is a risk that the orthosis may break if the load is too heavy. Max. permitted load: 200 kg. The air chambers must not be over inflated to prevent pressure points on the leg and to prevent the air chambers from bursting.

Purpose

Air Pump Walker®: Lower Leg Foot Orthosis for immobilisation in a given position

Air Pump Walker® Achill: This medical device is a lower leg foot orthosis for complete immobilisation of the lower leg and foot region in a defined, adjustable position

Indications / Contraindications

Indications

Air Pump Walker® / High shaft:

- Pre- and postoperative / post-traumatic immobilisation in case of damage to the bone / capsule and ligament structures of the foot / ankle
- Soft tissue injury
- Stable foot and / or ankle fractures
- Severe sprains grade II and III

Air Pump Walker® / Short shaft:

- Pre- and postoperative / post-traumatic immobilisation in case of damage to the capsule and ligament structures of the foot / ankle
- Soft tissue injury
- Stable foot fractures
- Sprains grade II

Air Pump Walker® Achill / High shaft:

- Achilles tendon ruptures (due to using Achilles Wedges)

Contraindications

- Unstable fractures
- Skin diseases in the body part treated
- Sensitivity disorders

Design Features

The Lower Leg Foot Orthosis is available in five sizes with a universal fit for right and left. The stable frame construction with perforations for air circulation achieves specific immobilisation of the lower leg / foot with air cushioning while mobilising the patient.

Use

1. Open the circular hook and loop fasteners and remove the front splint.
2. Put on the stocking included in the device packaging.
3. Open the blue liner and place the leg in the liner.
4. Close the liner all the way around. Ensure a wrinkle-free fit.
5. Position the front splint centrally on the blue liner. Ensure that the black pump is centred in the recess.
6. Guide the hook and loop straps through the loops. Pull the hook and loop straps as tight as necessary. Ensure that the leg is supported comfortably.
7. To achieve additional stabilisation of the ankle, the volume of the air chambers

can be increased using the pump attached to the side.

Note: The valve wheel must be turned in the "INFLATE" direction.

8. To reduce the volume, turn the valve wheel to the "DEFLATE" position.
This allows for individual filling and adjustment.

Toe Protector: For apical lesions or wounds on the toes, the toe protector can be removed or relocated.

Note

If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers using the valve. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

Use of the Achill version:

9. Open the hook and loop straps, the front splint and the lining.
10. Place the wedges in the heel area of the splint on the lining, corresponding to the degree of plantar flexion that is therapeutically required. A wedge has a height of approx. 10 mm. Depending on the number of wedges used, the height varies and the angle of plantar flexion can, thus, be adjusted individually.
11. Fix the wedges using the adhesive dots.
12. Depending on the course of therapy, the wedges can be removed and, thus, the height adjusted.
13. Once the wedges are positioned, carry out steps 2 to 7.

Combination with other products

Air Pump Walker® / High shaft: APWAW (Air Pump Walker® Achilles Wedges)

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

Sizes

Universal fit for right and left

Size	Rigid inner sole length	EU shoe sizes
XS	24 cm	34 – 36
S	26 cm	37 – 39
M	28 cm	40 – 42
L	30 cm	43 – 45
XL	32 cm	46 – 48

Colour

Grey-Blue

Material

Frame: PP

Outsole: TPR

Liner: PES, PU

Wedges: EVA

Stocking: Polyester, Cotton

Maintenance

The product does not require maintenance.

Care instructions

- Shell: Clean it with a damp cloth or soft brush.
- Lining / Stocking: Hand wash in cold water (30° C). Using mild soap. Do not bleach.
- Let the shell or liner dry in the air only. Do not dry the components on direct heat or sun.

Note

Pump can not be removed from the lining.

Disposal

Please dispose of the packaging in an environmentally friendly manner.

Dispose of this product after use in accordance with applicable local laws and regulations.

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Span Link International, LLC
CMC medical devices Ltd.
CMC medical devices GmbH