Toe Cap

- EN** Toe Cap
- DE** Zehenkappe
- ES** Puntera
- FR** Protège-orteils
- IT** Mascherina salvadita
- NL** Teenkap
- DA** Tåkappe
- SV** Tåkappa
- PL** Osłona na palce nóg
- SL** Kapica za nožne prste
- HR** Kapica za nožne prste
- PT** Proteção do dedo grande do pé

DARCO

Toe Cap

1

2

3

4

Toe Cap | Toe Cap

Important notes

Please always follow these instructions for use. In case of adverse effects (e. g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly.

DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

Purpose

The Toe Cap (photo 1) allows the DARCO orthopaedic device to be worn in closed form. It protects the foot from cold and dirt and improves comfortable fitting.

Use

Toe Cap for MedSurg Pro®, OrthoWegde Pro, Relief Dual® / Plus, TwinShoe®, Relief Insert® Walker 2.0 / Achill

1. Take out the removable insole of the orthopaedic device.
2. Remove the hook and loop fastener from the Toe Cap and now remove the white adhesive strip. (photo 2)
3. Then affix the hook and loop fastener parallel to the front edge of the sole onto the inner side of the outsole. Please note that the hook and loop fastener should not be affixed more than a maximum of 2 cm from the front edge of the sole. (photo 3)
4. Now slide the insole together with the Toe Cap into the shoe.
5. Press the insole gently with your hand so that it is fixed securely and close the shoe as usual. (photo 4)

Toe Cap for WCS®

1. Attach the supplied self-adhesive hook-and-loop strip to the outsole of the WCS®. Both ends should end approx. 3 – 4 cm before the heel counter.
2. Then position the toe cap centrally at the front end of the shoe and press it firmly along the sides (inside and outside).
3. To remove the toe cap, simply undo the hook-and-loop fastener. The self-adhesive strip remains on the outsole.

Working life / Lifespan of the product

The lifespan of the product is determined by natural wear and tear due to proper handling / cleaning.

Technical data

Sizes

Available in all corresponding shoe sizes for the products:

- Universal fit for right and left: MedSurg Pro®, OrthoWegde Pro, Relief Dual® / Plus, TwinShoe®, Relief Insert® Walker 2.0 / Achill
- Right and left version available: WCS®

Colours

Black: MedSurg Pro®, WCS®, OrthoWegde Pro, TwinShoe®

Grey: Relief Dual® / Plus, Relief Insert® Walker 2.0 / Achill

Materials

PU, polyester, nylon

Maintenance

The product does not require maintenance.

Care instructions

The DARCO Toe Cap must be washed by hand.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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