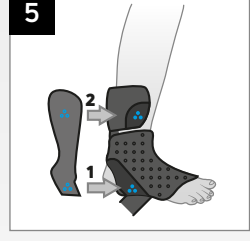
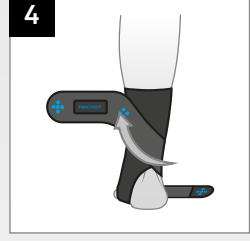
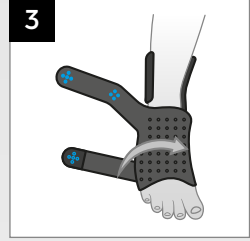
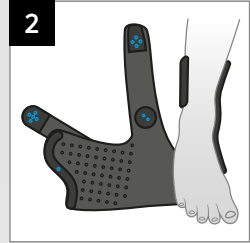
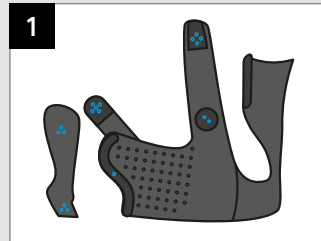
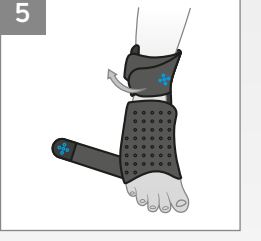
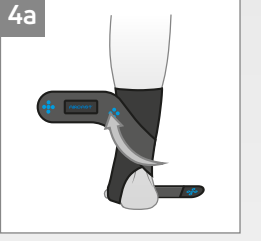
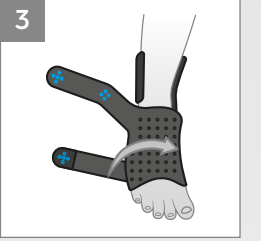
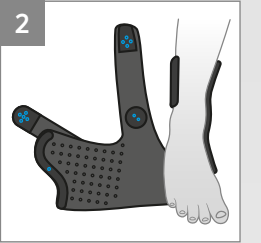
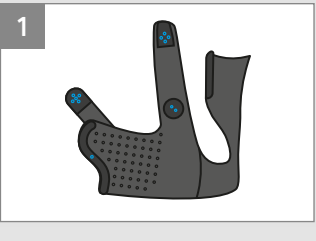
1

2

## DEUTSCH

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### VERWENDETE MATERIALIEN

Polyamid, Polyurethan, Acryl, Polyester, Elastan, Polypropylen, Haiz

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## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INTENDE USER/INDICATIONS

Airfree® is designed to provide support, stabilization and partial immobilization (restriction of movement) of the ankle.

- Acute lateral ligament injuries
- Acute ankle sprains and ligament ruptures
- Prevention of further injuries in pre-operative chronic ankle instability
- Protection of new lesions in case of instability chronic of the cheville

### Performance characteristics:

Providing support and stabilization to the limb or body segment.

Providing partial immobilization or controlled movement to the limb or body segment.

Providing mild protection of a limb or body segment for acute and prophylactic care.

### KONTRAINDICATIONS

- Do not use if you are allergic to any of the materials contained within this product
- Peripheral arterial diseases
- Impaired blood flow in the leg (e.g. with diabetes mellitus)
- Impaired lymphatic drainage or soft tissue swelling of unknown origin

### WARNINGS AND PRECAUTIONS

- The product is intended for single patient use only.
- We recommend that the first application be made under supervision of a health care professional.
- The frequency and duration of use should be determined by your health care professional.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not fit the Airfree® too tightly, as this could lead to local pressure symptoms.
- Do not apply to open wounds.
- If patient experiences any pain, swelling, sensation changes, or any unusual reactions while using this product, consult a health care professional immediately.
- It is not recommended to drive while wearing the brace as your ability to brake may be impaired.

**Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.**

### APPLICATION INSTRUCTIONS

The product is designed to be worn during normal everyday activities and sports. It can be worn over a sock.

For additional support and optimal stabilization, we recommend to wear a closed shoe with the brace. Hook and loop fasteners are marked with symbols to help fasten the straps in the correct position.

The footprint symbol on the inside of the brace indicates which ankle it goes in (left or right).

### 1 With the lateral stay

1. Unfasten the straps, remove the lateral stay and open the brace.
2. Slide foot into the back of the brace. The rigid part should be against the inside of the leg.
3. Fasten the hooks of the forefoot strap (▲) to the loops of the rigid part (▲).
4. Guide the calf strap around the back of the leg and fasten it to the top of the rigid part of the brace using the hook and loop fasteners (●).
5. Using the hook and loop fasteners, first attach the bottom part of the lateral stay (▲) to the bottom of the brace, followed by the top part of the lateral stay (▲) to the top of the brace.
6. Wrap the calf strap around the leg and fasten it to itself (●).
7. Pull the anti-supination strap diagonally across the instep and fasten it to the calf strap at the top of the brace (●).

The strap should be fastened tight enough to avoid supination movements.

### 2 Without the lateral stay

1. Unfasten the straps, remove the lateral stay and open the brace.
2. Slide foot into the back of the brace. The rigid part should be against the inside of the leg.
3. Fasten the hooks of the forefoot strap (▲) to the loops of the rigid part (▲).
4. Guide the calf strap around the back of the leg and fasten it to the top of the rigid part of the brace using the hook and loop fasteners (●).
- 4b. Wrap the rest of the calf strap around the leg and fasten it to itself (●).
5. Pull the anti-supination strap diagonally across the instep and fasten it to the calf strap at the top of the brace (●).

The strap should be fastened tight enough to avoid supination movements.

### CLEANING INSTRUCTIONS

Hand wash (30°C max) with mild detergent. Do not use if you are allergic to any of the materials contained within this product. Do not bleach. Enjugar a fondo. Dejar secar al aire. No secar con máquina secadora. No limpiar en seco. No planchar. No se pas repasser.

### COMPOSITION

Polyamide, polyurethane, acrylic, polyester, elastane, polypropylene, resin

**NOT MADE WITH NATURAL RUBBER LATEX**

**WARRANTY** D.J.O. LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**NOTE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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