EN

ONE SIZE FITS MOST/FITS LEFT OR RIGHT

This bandage provides protection for the ankle and helps ease painful, tired feet and swollen ankles during daily activities and sports.

Made of Neoprene and breathable spacer fabric with Velcro fastener that allows optimal individual fitting.

Indications:

Suitable for the treatment of ankle sprain, other ankle injuries, ankle pain and foot pain.

Useful as support for strained and sprained ligaments.

Fitting Instructions:

1. Wrap the strap around the back of your ankle.
2. Bring the bandage underneath your foot then wrap it around your ankle (as shown).

Caution:

Use the product for the mentioned indication only. For single person use only. Do not use the product if you are allergic to one of its components. If you experience pain, swelling, or sensation changes as a result of wearing this item, stop use immediately and consult your healthcare provider. Avoid tight application as it could restrict circulation.

Material Composition: 80% Neoprene, 20% Nylon.

Composition of the spacer fabric is as follows: 55% Spandex, 45% Polyester.

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URIEL IS A TRADE NAME OF MEDITEX GROUP LTD.