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# Instruction for use – Catell Armsling Standard


Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports the users arm and shoulder after injury or surgery.

## Application

1. Open the arm pouches hook and loop closures and place the forearm in the pouch. Place the strap with plastic buckles at wrist level. Make sure part with plastic buckle end up facing towards body. Close the strap. Picture 1
2. Place and close the wide short hook strap over the opening of the pouch, near the bend of the arm. Picture 2
3. Pull the strap fixed on the pouch behind your back, over the opposite shoulder, turn through the plastic buckle and attach the hook on the same strap. Thoroughly adjust the correct length of strap for optimal support. Picture 3
4. In need of additional support for your hand, place the short narrow strap in the thumb webbing, between thumb and index finger, and attach to the arm pouch. Picture 4a
5. In need of extra stability of the arm towards body, the accompanying band is used as a waist belt. Attach the belt to arm pouch at elbow side, place it around your back and turn through plastic buckle at wrist level. Picture 4a, 4b
6. Make sure that all straps are properly secured.

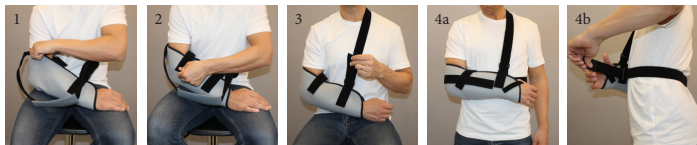When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation

## Materials

Nylon, polyester. Free from latex.

## Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

## Size

Size	*Dimensions	Size	*Dimensions	Size	*Dimensions	Size	*Dimensions	Size	*Dimensions
XS	23 - 28 cm	S	28 - 33 cm	M	33 - 38 cm	L	38 - 43 cm	XL	43 - 48 cm

\*Measure length of forearm. Measurements should be seen as guidelines. The product is bilateral. Single patient use.

## Recycling

Separate plastic and textile. Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örskroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

240416:4