TheraPIE Knete

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Instructions for use

Intended use:

This medical device is a latex-free therapeutic plasticine which is used to improve movement and function of the neuromuscular circuit through special finger and hand exercises.

Indications:

Restoration of the ability to move the hand

Contraindications:

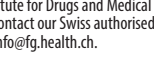
Diseases of the skin in the palm area, e.g.: (neurodermatitis, open wounds).

Risks of use / Important information


- Use TheraPIE Knete according to the indication.
- simultaneous use with other products only after consultation with your doctor or therapist
- do not make any changes to the product
- do not use on open wounds in the palm area
- do not reuse - this aid is intended for the care of a patient
- Caution: not suitable for consumption
- Not suitable for children under 3 years of age
- Do not get on clothing or other textiles or similar
- If plasticine has been put in the eye, wash the eye out with water. Consult a doctor if necessary
- In case of pain, redness, swelling or other problems, reduce the exercise frequency or resistance or stop the entire exercise programme.

Instructions for use

Discuss the use and duration with your prescribing doctor. The selection of the appropriate strength and instruction will be given by the specialist personnel from whom you received the medical device.

Wrist flexion

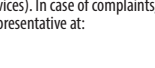Fingertip grip

Finger extension

Palm ball rolls

Storage

Due to the viscous flow behaviour of the therapeutic plasticine, as well as against contamination, the therapeutic plasticine should always be stored in the closed tin, at room temperature.

Material composition

Silicone oil (82.00%), silicon dioxide (15.00%), boric acid (2.00%), colour pigments (1.00%).

Cleaning instructions:

The therapeutic plasticine cannot be cleaned. Please wash and disinfect your hands thoroughly before use.

Warranty

The legal regulations of the country in which you purchased the product apply to the purchased product. Please contact your dealer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been sufficiently observed, the warranty may be impaired or invalidated. A warranty is excluded in the case of use not in accordance with the indication, non-observance of the application risks and instructions as well as unauthorised changes to the product

Useful life / service life of the product

The service life of TheraPIE Knete is determined by natural wear and tear when handled properly and in accordance with the instructions for use.

Obligation to report

If there is a serious deterioration in the state of health when using the medical device, report this to your specialist dealer or to us as the manufacturer and to the BfArM (Federal Institute for Drugs and Medical Devices). In case of complaints, contact our Swiss authorised representative at: info@fg.health.ch.

Disposal:

At the end of use, the product must be disposed of in accordance with local regulations.

Declaration of conformity

We confirm that this product complies with the requirements of REGULATION (EU) 2017/745 of the European Parliament and of the Council. The current declaration of conformity can be found at the following link: www.premium-therapie.de/konformitaet
Status: 03.2022

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