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Art.-Nr. 11209 000 weiß
Art.-Nr. 11209 100 Buche Dekor

PZN 00828437
PZN 00828420

PHV-Nr. 50.45.04.0011

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PZN 00828437
PZN 00828420

PHV-Nr. 50.45.04.0011

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

The »Basic« side table is designed to be used in the domestic environment and provides valuable support in the daily administration of care for those who are in need of it, the disabled and the frail, who are predominantly bedridden.

It makes eating and drinking in bed easier, for example, when the person in need of care is unable to take their food independently out of bed, and also independent personal hygiene in bed.

The »Basic« side table is suitable for repeated re-use. Before re-using the following measures are to be undertaken by a specialist shop:

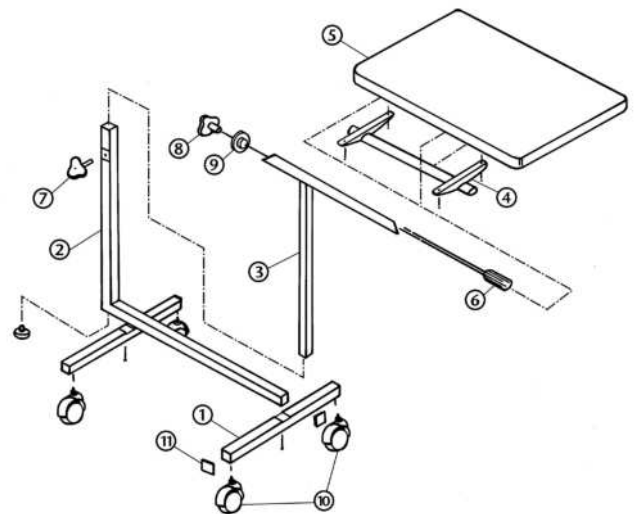
1. Checking the general condition, wear and tear, damage and functionality and undertaking of repairs if necessary.
2. Disinfection by wiping, taking into consideration the disinfection instructions in the Cleaning section.

⚠ Safety notice ⚠

- Make sure of the functionality, stability and proper condition of the side table and accessories before use.
- Regularly check the stability and firmness of the product and if possible make sure the screws are tight.
- Always consider the maximum central loadbearing capacity of the side table!
- Do not lean on the side table.
- Always apply the brakes to the castors while stationary.
- Children should only use the table under adult supervision - danger of trapping and tipping!

Delivery circumference:

1. Base frame
2. L-pole
3. Retaining tube for the table top (arranged inside)
4. Fixing pole for the table top
5. Table top
6. Iron branch pole with a bevelled edge
7. Winged screw with thread (for adjustment of height)
8. Winged screw (for adjustment of tilt)
9. Metal disk
10. 4 castors with brakes
11. Plastic plugs

Assembly

Attach the two legs (1), which are each provided with 2 castors (10), with 2 hexagon head screws under the base of the L-pole. Take care that the legs are attached at the correct marked position (A to A, B to B, C to C and D to D).

Place the retaining pole for the table top (3) in the vertical part of the L-pole (2), whereby the axis of the pole must lie above the base piece provided with the castors.

Screw the black winged screw with thread (7) intended for height adjustment into the hole of the L-pole.

Release the metal disk (9) from the steel branch pole (6) and place this pole in the retaining pole (3) ensuring that the slanted head fits exactly to the slanted end and the pole is therefore closed.

Place the table top (5) horizontally onto the retaining pole (3) with the fixing pole (4) ensuring that the steel branch pole (6) with the thread protrudes on the side. Insert the metal disk (9) here and place the black winged screw (8) on the end and tighten.

Adjusting the side tables

1. Positioning the table top

In order to change the tilt angle turn the black winged screw at the side with one hand and adjust the desired tilt of the table top with the other.

2. Adjusting the height of the table top

The height of the table top can be adjusted between 70 and 116.5 cm. To do this, loosen the winged screw by turning it to the left. Move the central pole up or down to the desired height and fix in position by tightening the screw clockwise.

3. Fixing the stationary position

You can park the side table by pressing the roll brake of the castors down using your foot. Release the brake by pressing the brake lever upwards.

Cleaning

A routine wipe clean with a (damp) cloth is generally enough.

Clean the side table with a mild household cleaner. Do not use any caustic or abrasive cleaning agents.

For surface-wipe disinfection use a standard, gentle, DGHM-/VAH¹-listed disinfectant (take note of the exposure time). When selecting the disinfectant make sure that it is mild (kind to the skin and surfaces) and environmentally friendly.

Material

- Column and base: steel
- Table top: wooden board with a plastic frame
- Castors/adjustment buttons: steel with a plastic coating

Technical Details

- Table top dimensions: 40 x 60 cm
- Total weight: 8.5 kg
- Maximum, central loadbearing capacity in a horizontal position: 10 kg
- Height adjustment: 70-116.5 cm

¹ DGHM: German Society for Hygiene and Microbiology

VAH: German Society for Veterinary Medicine

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