Finger flexion splint
Férula de flexión de dedo
IF601

Finger extension splint
Férula de extensión de dedo
IF501

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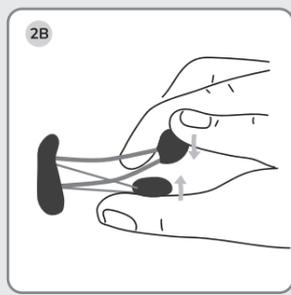
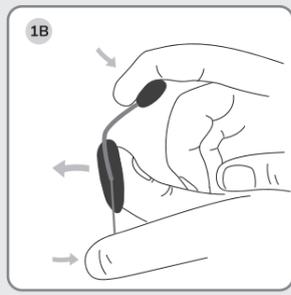
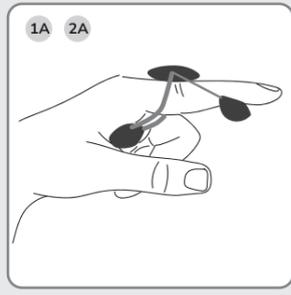
EN IF501 Finger extension splint IF601 Finger flexion splint... Dear customer, we thank you for the trust you have placed in an EMO product. By choosing our line of finger orthoses you have acquired a range of quality products with a high sanitary level... is essential for the product to function properly and to reduce the risk of injury or re-injury inherent in the use of any product... CONTRAINDICATIONS: Do not use directly on open or red scars without being properly protected... PRECAUTIONS AND WARNINGS: Do not use directly on open or red scars without being properly protected... WASHING, MAINTENANCE AND STORAGE: The product can be cleaned and stored... PRODUCT LIFE / MANUFACTURER WARRANTY: ESPECIALIDADES MÉDICO-ORTOPÉDICAS, S.L (EMO) guarantees all its products as long as have not been manipulated or altered and always within the framework and deadlines of the legal provisions... QUALITY CERTIFICATES: The materials used in the manufacture have been tested and meet the requirements of the applicable European regulations... WARNING: Do not use directly on open or red scars without being properly protected... IMPORTANT INFORMATION: Please read all warnings and instructions before using the product.

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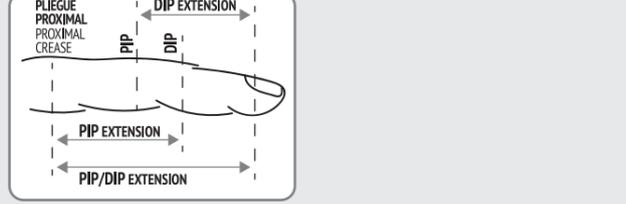
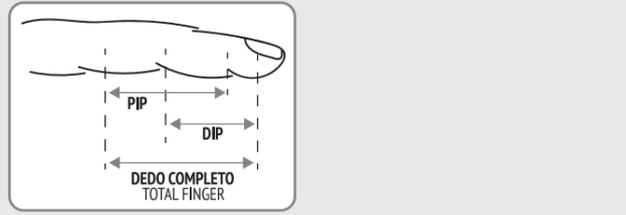
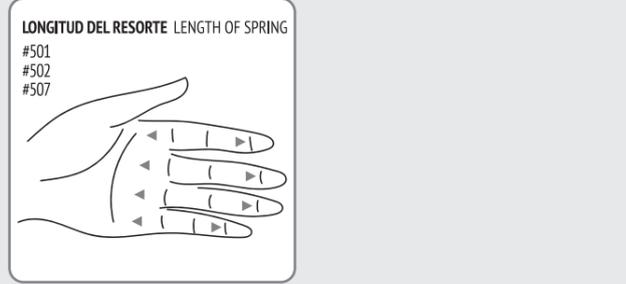
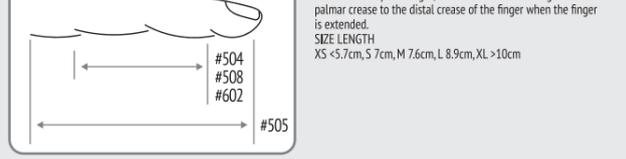
INSTRUCTIONS FOR USE AND PLACEMENT: SPRING ASSEMBLY (A) 1. Hold the spring by the proximal and distal pads. NOTE: The proximal pad has the white plastic covered pad has the white plastic covered wire attached to it. 2. Slide the splint onto the finger so that the proximal pad rests on the volar aspect of the metacarpal head and the distal pad rests on the distal interphalangeal (DIP) joint. ADJUST THE SPRING TENSION (B) 1. To increase extension force, spread the volar pads apart before placing them on the finger. 2. To decrease the force of extension, squeeze the volar pads together before placing it on the finger.

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SPRINT LENGTH / SIZE SELECTION To determine splint length measure the affected finger from the palmar crease to the distal crease of the finger when the finger is extended. SIZE LENGTH XS <5.7cm, S 7cm, M 7.6cm, L 8.9cm, XL >10cm