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ITEM T25

URIEL IS A TRADE NAME OF MEDITEX GROUP LTD.

www.uriel.eu
www.facebook.com/UrielMeditexGroupLtd

WRIST SUPPORT

Dear Customer,

Thank you for purchasing this product. URIEL is dedicated to offering comfortable and easy to use products, using the finest materials and most advanced production techniques, to provide our customers with the best solutions in the market. To fully enjoy the benefits of this product, please read the below information.

Intended use

This Wrist Support is made of thick THERMAL Neoprene, with 100% cotton towel lining. It helps retain body heat promoting faster healing and relieving pain. Takes the strain off the wrist and provides firm support whilst allowing for normal daily function. Supports the wrist joint and prevents injury to wrist.

Fitting Instructions

1. Put the brace on your hand with the thumb inserted in the thumb hole
2. Wrap around your wrist and close with the hook and loop fasteners for optimum support and comfort

Caution

Use the product for the mentioned indication only. For single person use only. Do not use the product if you are allergic to one of its components. If you experience pain, swelling, or sensation changes as a result of wearing this item, stop use immediately and consult your healthcare provider.

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