INSTRUCTIONS FOR USE

A101

ANKLE BRACE / LACE

INTENDED USE

Best for helping to prevent and/or recover from common ankle sprains.

SIZE

SIZE:	2XS	XS	S	M	L	XL
Inch	-7.5	7.5-8	8-8.5	8.5-9	9-9.5	9.5-10
Cm	-19.1	19.1-20.3	20.3-21.6	21.6-22.9	22.9-24.1	24.1-25.4

WARNINGS & PRECAUTIONS

This product is not a substitute for medical care. Always seek professional medical advice for the diagnosis and treatment of pain, injury or irritation. Neoprene supports are intended for use during short periods (several hours) of activity. Do not wear for more than 2-3 hours. Do not wear while sleeping. While being worn, especially during initial usage, regularly inspect the skin under the support. If a rash or other skin problems develop, discontinue use. If the problem persists, consult a physician. Do not apply over open wounds or if you have a susceptibility to dermatitis, an allergy to rubber or circulation problems

DIRECTIONS

Self Explanatory.

CONTENTS

The product is provided nonsterile and is intended to be reusable on the same person multiple times. The product is manufactured out of 52% PVC/46% polyester/2% rubber.

Front elastic band: 63% Polyester/ 37% Rubber (with Natural Rubber Latex).

 ⚠️ **CONTAINS LATEX:** This product contains natural rubber latex, which may cause allergic reactions.

If any skin irritation occurs, discontinue use and consult a physician.

CLEANING

 Hand wash in warm water with a mild detergent, rinse thoroughly and drip dry. **DO NOT MACHINE DRY.**
STORAGE

When not in use, store in a cool, dry place.

EN

FR

DE

IT

ES

PT

NL

BG

HR

CS

DA

ET

FI

EL

HU

LV

LT

PL

RO

SK

SL

SV

NO

INSTRUCTIONS FOR USE

A101

ANKLE BRACE / LACE

DISPOSAL

Check product for excessive wear prior to each use. Discard the device if it shows signs of excessive wear and replace with a new product as necessary.

WHEN TO CONSULT A HEALTHCARE PROFESSIONAL

Consult a healthcare professional if experiencing any adverse reaction to the device including skin reaction or changes in condition.

NOTICE TO USER AND/OR PATIENT

In the event of an adverse incident related to the device, remove the device and report the incident to the manufacturer.

LABEL SYMBOLS

The symbols used on the product label or product literature are defined in the Symbols Legend here: www.usb-eifu.eu/symbols

 United Sports Brands, Inc.
11488 Slater Ave.
Fountain Valley, CA 92708 USA
For Info:
www.McDavidUSA.com
+1-800-233-6956

 United Sports Brands
Janssen-Pharmaceuticaan
Post Box 4
2440 Geel, Belgium
For Info: www.McDavid.eu

 CH Imported by:
United Sports Brands
Switzerland GmbH
Westbahnhofstrasse 1
CH-4500 Solothurn, Switzerland
+32 (0) 14 85 47 85

EC	REP
----	-----

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QNET B.V.
QNET CH-REP GmbH,
QNET LTD.