## Rolyan® Adjustable Outrigger Kit for Extension:

A601-1

## Rolyan® Individual Units for Flexion and Extension:

A601-2

## Rolyan® Preformed Adjustable Outrigger for Extension:

A601-71 through 78

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed. Figures referred to in the instructions are on page 8.

### INDICATIONS

- To increase extension of finger joints

### Individual Units for Flexion and Extension

- Also to increase flexion of finger joints

### CONTRAINDICATIONS

- Not for use with fixed bony contractures
- Not for immediate treatment following fractures

### PRECAUTIONS

- These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the straps are not so tight that they interfere with circulation.
- When cutting metal end, hold or cover piece to be removed to prevent it from flying.

### Preformed Adjustable Outrigger

- This product has components that contain natural rubber latex, which may cause allergic reactions in some individuals.
- After heating splinting material, always check its temperature before applying it to the patient.

### CARE AND CLEANING

- All components can be cleaned with soap and lukewarm water. The straps can be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

### Preformed Adjustable Outrigger

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.

### INSTRUCTIONS FOR USE

Directions and photographs show splints used to increase extension; the instructions also apply to individual units that are used to increase flexion, with the splint base and components applied to the volar surface.

### Adjustable Outrigger Kit

#### Preformed Adjustable Outrigger

- Evaluate outrigger for fit—it should extend beyond radial and ulnar borders of hand.

#### Adjustable Outrigger Kit

#### Individual Units for Flexion and Extension

- Fabricate appropriate splint base.

- Position outrigger on splint and mark position (A).
- Cut piece of low-temperature thermoplastic large enough to cover end sections of outrigger (B).
- If splint base and overlapping piece are coated, remove coating by scraping surface or brushing it with solvent.
- Heat end sections of outrigger with heat gun and embed into splint base, making sure ends do not poke through splinting material.
- Heat overlapping piece of splinting material, make sure it is hot and dry, and place it over end sections of outrigger to bond them to splint base.
- Attach straps to splint as appropriate.
- With self-adhesive loop on surface of splint that will be toward skin, wrap D-ring rubber band attachment strap around proximal end of splint. Position strap so D-ring is on outside surface of splint (C).
- Attach rod adjuster(s) loosely to outrigger by slipping slot in rod adjuster over outrigger wire and tightening set screws with Allen wrench (D).

### Preformed Adjustable Outrigger

- If modifications are necessary, they should be made before attaching Plastazote® lining. To modify splint, which is made from ¼" (3.2mm) Polyform® splinting material, spot-heat it with a heat gun or in a heat pan of water at a temperature of 150° to 160°F (65°–70°C) until pliable. **Heating entire splint may cause it to lose its preformed shape.** Do not heat splint where outrigger is attached.
- Remove backing from self-adhesive strips on foam lining and attach lining to interior of splint.
- Attach straps as follows:
  - Palmar strap—attach one self-adhesive hook coin to palmar bar and one to ulnar side of splint. Attach small strap.
  - Proximal strap—use strap with self-adhesive hook tab.
  - Wrist strap—attach rectangular hook piece to dorsum of splint proximal to outrigger. Attach a hook coin to each side of splint. Attach center of long strap to rectangular hook piece, cross strap ends on volar surface of wrist, and attach to hook coins (E).

- Pull lining away from proximal end of splint. With self-adhesive loop on surface of splint that will be toward skin, wrap D-ring rubber band attachment strap around proximal end of splint. Position strap so D-ring is on outside surface of splint (C).

### Adjustable Outrigger Kit

#### Individual Units for Flexion and Extension

#### Preformed Adjustable Outrigger

- Insert an extender rod through each rod adjuster. Adjust position of rod adjuster so extender rod lies parallel to joint being splinted. Use Allen wrench to tighten screw on rod adjuster to hold extender rod in place (F).

- To determine final position of extender rod, use Allen wrench to provide a 90° angle relative to the finger (G).
- Tighten the two set screws that hold rod adjuster on outrigger alternately, applying even tension to each screw.
- Remove splint from patient and secure all set screws.
- Using heavy-duty wire snips, cut off proximal end of extender rod. Extender rod should extend at least ½" (1.3 cm) beyond rod adjuster. Cover end with black tip protector.
- Attach appropriate rubber bands or springs to proximal plastic D-ring (C).

**Preformed Adjustable Outrigger:** Number 14 rubber bands and gold (size F) graded springs are included. The traction component that is most appropriate can be used.

Loaded heights of size F 2" (5.08cm) graded spring are as follows:

2.25" (5.72 cm)	8.8 oz. (250 g)
2.5" (6.35 cm)	11.5 oz. (327 g)
2.75" (6.99 cm)	14.4 oz. (408 g)
3" (7.62 cm)	17 oz. (481 g)
3.25" (8.26 cm)	19.8 oz. (562 g)
3.5" (8.89 cm)	22.6 oz. (640 g)
3.75" (9.53 cm)	25.3 oz. (717 g)
4" (10.16 cm)	28 oz. (794 g)

- With splint on extremity, position finger loop or sling on finger and put monofilament into curled end of extender rod.
- Attach spring or rubber band that is on proximal plastic D-ring through large hole in aluminum connecting piece on finger loop or sling.
- Adjust length of monofilament line to provide appropriate force of traction by sliding monofilament through two small holes on aluminum connecting piece (H). To increase amount of traction force, reduce length of monofilament. To reduce amount of traction force, increase length of monofilament
- For thumb extension or to apply a force couple following MCP arthroplasty, attach a rod adjuster to radial side of curved portion of outrigger. Slide extender rod into rod adjuster and bend it to position finger sling to provide appropriate angle of pull (I). **Preformed Adjustable Outrigger:** thumb sling with number 10 rubber band is included.

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**Patterson Medical Ltd.**