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Silopad Arthritic/Diabetic Gel Sock is a single layer sock designed with 1/16" gel throughout the sole. The gel insole provides added support, cushioning, protection, and comfort while reducing friction, shear forces, and callus formation. A seamless toe design eliminates abrasion and pressure on the toes, while the non-restrictive cuff allows for optimum circulation. The Arthritic/Diabetic sock can be worn with or without diabetic shoes. This product is washable and reusable. • Helps reduce friction, abrasion, shear forces, and callus formation. • Serves as a soft tissue supplement to fat pad atrophy. • Recommended for diabetic, rheumatoid, and the insensate foot.

DIRECTIONS & USE Hand Wash. Air Dry. 20 Hours of Wear. Do Not Bleach.

WARNINGS If you have diabetes or poor circulation, consult your physician before using this product. If pain, numbness, discoloration, or irritation occurs, discontinue use immediately. If symptoms persist, consult a medical professional. Do not place product over open wounds. Keep out of reach of children and pets. Latex free & "Not made with natural rubber latex" Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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© Silipos Holding, LLC