MEASUREMENTS		
SIZE	IN	CM
S-XL	30 - 46	75 - 115

REF AC07

ONE SIZE

Qty.1

ACTIVE

REF AC07

Lumbo Belt

FASTENERS

BREATHABLE

ONE SIZE

REF AC07

LUMBO BELT

ONE SIZE

To maintain the product quality, we reserve the right to substitute the material without prior notice.
 URIEL IS A TRADE NAME OF MEDITEX GROUP LTD.
 WW.URIEL.EU
 5702 Niedenz Switzerland
 Lenzburgerstrasse 2
 CH REF Orthopartner AG
 48163 Muenster Germany.
 Borkstrasse 10,
 EC REF MedNet EC-REP GmbH
 3688523 NESHER, ISRAEL.
 MEDITEX GROUP Ltd.

635AC070

Caution: Use the product for the mentioned indication only. For single person use only. Do not use the product if you are allergic to one of its components. If you experience pain, swelling, or sensation changes as a result of wearing this item, stop use immediately and consult your healthcare provider. Avoid tight application as it could restrict circulation.

Material Composition: 80% Neoprene, 20% Nylon.
 Composition of the spacer fabric is as follows: 55% Spandex, 45% Polyester.

ONE SIZE FITS MOST

Provides mechanical support by reducing lumbar pressure and providing a compression effect around the lower back and abdomen. Recommended for orthopedic conditions including lumbar facet joint syndrome, sciatica, spondylitis, and post-surgical lumbar procedures. Made of Neoprene and breathable spacer fabric with hook and loop fastener that allows optimal individual fitting.

Indications: Recommended for Lumbar Facet Syndrome, lower back pain, Sciatica, Osteoarthritis, moderate form of Spinal Muscular Atrophy, post-surgery management.

Fitting Instructions: Place the belt around your lower back, as shown in the picture, position in the desired height and fasten the Velcro fasteners in a firm yet comfortable fit.

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