REF AC27L

MEASUREMENTS		
SIZE	IN	CM
S-XL	5.5 - 9.5	14 - 22

ONE
SIZE

Qty.1

ACTIVE

L
LEFT

AC27L WRIST SPLINT

LEFT SIDE

AC27L WRIST SPLINT

ONE
SIZE

To maintain the product quality, we reserve the right to substitute the material without prior notice

635AC27L

CH REP Orthopartner AG
Lenzburgstrasse 2
5702 Niederlenz Switzerland

EC REP MedNet EC-REP GmbH
Borkstrasse 10,
48163 Muenster Germany.

MD MedNet EC-REP GmbH
23, Hamessilla Str.,
3688523 NESHER, ISRAEL.

MD MEDITEX GROUP Ltd.
3688523 NESHER, ISRAEL.

Material Composition: 80% Neoprene, 20% Nylon.
Composition of the spacer fabric is as follows:
55% Spandex, 45% Polyester.

Caution: Use the product for the mentioned indication only.
For single person use only. Do not use the product if you are
allergic to one of its components. If you experience pain,
swelling, or sensation changes as a result of wearing this item,
stop use immediately and consult your healthcare provider.
Avoid tight application as it could restrict circulation.

Fitting instructions:

1. Insert your hand into the splint with your thumb protruding through the hole as shown in the picture.
2. Close and fasten the wrist band.
3. Close and fasten the thumb band.

Indications: Recommended for the treatment of wrist pain due to strain, sprain, fracture, inflammation or carpal tunnel syndrome. For use as a transition brace after cast removal, or treatment of ligament, tendon or muscle injuries.

Provides firm wrist support while allowing for normal daily functions. With an aluminum palmar stay designed to provide firm wrist support by holding the wrist in a functional position at an extension of 35 degrees (adjustable, optional).
Made of Neoprene and breathable spacer fabric with Velcro fastener that allows optimal individual fitting.

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