## Ankle Bandage

REF AC37

Ver.SW-AC37-250916

www.uriel-global.com

15815 SW 11th Court Rd. Ocala,  
Florida 34473 USA  
FDAbasics LLC

5702 Niederenz Switzerland

Lenzburgerstrasse 2  
Orthopartner AG

48163 Muenster Germany.

Borkstrasse 10,  
MedNet EC-REP GmbH

3 Hazayit st., P.O. 413  
Mevo Carmel Industrial Park, Israel.  
MEDITEX GROUP Ltd.

### Ankle Bandage

This bandage provides protection for the ankle and helps ease painful, tired feet and swollen ankles during daily activities and sports.

Made of Neoprene and breathable spacer fabric with Velcro fastener that allows optimal individual fitting. ONE SIZE FITS MOST/FITS LEFT OR RIGHT.

This product is recommended for use by individuals aged 18 and above only. For home, outpatient or institutional use. Single Patient Multiple Use. Medical Device.

### Medical Indication

Suitable for the treatment of ankle sprain, other ankle injuries, ankle pain and foot pain. Useful as support for strained and sprained ligaments.

### Caution

Consult your physician if you encounter severe sideeffects or discomfort. Use product for the mentioned indications only. For single patient use only. For external use only. Do not use product if you have allergy to one of it's components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider. Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### Fitting Instruction

Wrap the strap around the back of your ankle. Bring the bandage underneath your foot then wrap it around your ankle (as shown).

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

### Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

### Material Composition

80% Neoprene, 20% Nylon. Composition of the spacer fabric is as follows: 55% Spandex, 45% Polyester.

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