Carpal Tunnel Night Mitten

REF 213CTS

Ver.SW-213CTS-251113

www.uriel-global.com

US Agent
FDAbasics LLC
15815 SW 11th Court Rd. Ocala,
Florida 34773 USA

CH REP
Orthopartner AG
Lenzburgerstrasse 2
5702 Niederlenz Switzerland

48163 Muenster Germany.

EU REP
MedNet EC-REP GmbH
Borkstrasse 10,

3 Hazayit st., P.O. 413
Mevo Carmel Industrial Park, Israel.
MEDITEX GROUP Ltd.

Carpal Tunnel Night Mitten

Specifically designed to provide relief for those suffering from Carpal Tunnel Syndrome (CTS) pain at night. It alleviates nighttime symptoms such as pain and finger numbness by maintaining the fingers in a relaxed bent position. Helpsto reduce pressure on the median nerve and improves blood flow to the nerve. Made from breathable fabric, the glove ensures ventilation and comfort for extended use during sleep.

Indications

CTS-related pain and numbness in the fingers caused by Carpal Tunnel Syndrome.

Contraindications

Arthritis or arthritis in the finger joints.

Fitting Instruction

Use mittens during sleeping hours mainly. Select size using chart on the packaging. Slide your hand into the mitten, secure velcro strap around your wrist, and bend your fingertips to fasten additional velcro. Ensure velcro strap around wrist is not too tight. If discomfort occurs, adjust strap or fit.

Caution

Use the product for the mentioned indication only. For single person use only. Do not use the product if you are allergic to one of its components. If you experience pain, swelling, or sensation changes as a result of wearing this item, stop use immediately and consult your healthcare provider. Avoid tight application as it could restrict circulation.

Material Composition

Fabric: 80% Nylon+20% Spandex. Yellow Stripes: 100% Silicone.

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