Finger Splint

REF 238

Ver.SW-238-250916

www.uriel-global.com

US Agent
FDAbasics LLC
15815 SW 11th Court Rd. Ocala,
Florida 34473 USA

CH REP
Orthopartner AG
Lenzburgerstrasse 2
5702 Niederlenz Switzerland

EU REP
MedNet EC-REP GmbH
Borkstrasse 10,
48163 Muenster Germany.
MEDITEX GROUP Ltd.
3 Hazayit st., P.O. 413
Mevo Carmel Industrial Park, Israel.

ENGLISH

Finger Splint

This product is designed to immobilize the injured finger. Lightweight flexible PE shell and EVA pad hold the finger firmly, providing good support and protection. With hook and loop strap for optimal fitting.

This product is recommended for use by individuals aged 8 and above only. For home, outpatient or institutional use. Single Patient Multiple Use. Medical Device.

Medical Indication
Broken finger, finger injuries.

Caution
Consult your physician if you encounter severe sideeffects or discomfort. Use product for the mentioned indications only. For single patient use only. For external use only. Do not use product if you have allergy to one of it's components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Fitting Instruction
Fit around injured finger as shown in the picture. Fasten the hook and loop strap tightly yet comfortably.

Hand wash only. Do not expose to temperatures above 35°C / 95°F.

Storage and Transportation
Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

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