## Hallux Valgus Post Surgery Bandage

REF 38

Ver.SW-38-250916

www.uriel-global.com

15815 SW 11th Court Rd, Ocala,  
Florida 34473 USA

US Agent FDAbasics LLC

5702 Niederenz Switzerland

CH REP Orthopartner AG  
Lenzburgerstrasse 2

48163 Muenster Germany.

EU REP MedNet EC-REP GmbH  
Borkstrasse 10,

3 Hazayit st., P.O. 413  
Mevo Carmel Industrial Park, Israel.  
MEDITEX GROUP Ltd.

Hallux Valgus Post Surgery Bandage  
Suitable for both feet - One size fit all.

This product is recommended for use by individuals aged 18 and above only. For home, outpatient or institutional use. Single Patient Multiple Use. Medical Device.

### Medical Indication

After Hallux Valgus surgery it is recommended to wear the bandage whenever possible to help to reduce pain.

### Caution

Consult your physician if you encounter severe sideeffects or discomfort. Use product for the mentioned indications only. For single patient use only. For external use only. Do not use product if you have allergy to one of it's components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider. Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### Fitting Instruction

Open all fasteners. Open the bandage on the floor with the smooth side facing down. Place your foot on the bandage so that your big toe is positioned in the middle of the two vertical lines. Close the fastener over your toe. If necessary rotate the bandage so that the supporting stay is facing outwards. Close and fasten the rest of the bandage so that the logo is facing up as shown in the photograph.

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

### Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

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