Rib Belt

REF 15M

Ver.SW-15M-250916

www.uriel-global.com

US Agent
FDAbasics LLC
15815 SW 11th Court Rd. Ocala,
Florida 34473 USA

CH REP Orthopartner AG
Lenzburgerstrasse 2
5702 Niederenz Switzerland

EU REP MedNet EC-REP GmbH
Borkstrasse 10,
48163 Muenster Germany.
MEDITEX GROUP Ltd.
3 Hazayit st., P.O. 413
Mevoe Carmel Industrial Park, Israel.

Rib Belt

This brace provides firm support for aching chest muscles, fractured ribs or thorax injuries. Strong laminated foam material, lined with soft cotton terry fabric that provides comfort and sweat management. Easy adjustment with hook and loop fastener.

This product is recommended for use by individuals aged 18 and above only. For home, outpatient or institutional use. Single Patient Multiple Use. Medical Device.

Medical Indication

Rib fractures, chest injuries or trauma, after thoracic surgery, chronic ribs pain, support for rehabilitation and mobility.

Caution

Consult your physician if you have severe postural issues or spinal conditions. Use the product for the mentioned indications only. For single patient use only. For external use only. Do not use the product if you have an allergy to one of its components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Fitting Instruction

Place the belt around your chest as shown in the picture, position in the desired height and fasten the hook and loop fasteners in a firm yet comfortable fit.

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

Material Composition

Foam Fabric: 40% Cotton, 40% Polyester, 10% Polyurethane foam, 10% Elastane.
Elastic Fabric: 52% Polyamide, 22% Elastane, 16% Latex, 10% Polyester.
Contains natural rubber latex.

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