Arm / Shoulder Bandage

REF 86U

Ver.SW-86U-250916

MEDITECH GROUP Ltd.
3 Hazayit st., P.O. 413
Mevoe Carmel Industrial Park, Israel.
EU REP MedNet EC-REP GmbH
Borkstrasse 10,
48163 Muenster Germany.
CH REP Orthopartner AG
Lenzburgstrasse 2
5702 Niederlenz Switzerland
US Agent FDAbasics LLC
15815 SW 11th Court Rd. Ocala,
Florida 34473 USA
www.uriel-global.com

ENGLISH

Arm / Shoulder Bandage

For treatment of traumatic or post-surgical shoulder immobilization. Provides excellent stabilization of the shoulder while allowing free movement of the fingers. Maximum wearing comfort.

This product is recommended for use by individuals aged 18 and above only.

For home, outpatient or institutional use.
Single Patient Multiple Use. Medical Device.

Medical Indication

For immobilization and support in cases of shoulder, arm, or forearm fractures, shoulder post-dislocation/subluxation, rotator cuff injuries, pre- and post-operation care for shoulder, arm, or forearm procedures; pain resulting from soft tissue inflammation, including tendinitis, ligament sprains, and muscle strains.

Caution

Consult your physician if you have severe postural issues or spinal conditions. Use the product for the mentioned indications only. For single patient use only. For external use only. Do not use the product if you have an allergy to one of its components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Fitting Instruction

Remove and discard the covers on the fasteners. Open all fasteners. Insert your arm into the sleeve with your thumb extending from the thumb hole. Pass the strap attached to the upper part of the sleeve over your shoulder, loop it around your wrist and fasten (as shown in the picture). Pass the other strap around your back, make a loop above your elbow and fasten (as pictured). Adjust the straps for comfort and firmness.

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

Material Composition

Sleeve: 90% Cotton, 10% Polyester Coated Rubber.
Strap: 40% Cotton, 40% Polyester, 10% Polyurethane foam, 10% Elastane.

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