Wrist Bandage

REF ST20

Ver.SW-ST20-250916

www.uriel-global.com

US Agent
FDAbasics LLC
15815 SW 11th Court Rd.Ocala,
Florida 34473 USA

5702 Niederlenz Switzerland

CH REP
Orthopartner AG
Lenzburgerstrasse 2

48163 Muenster Germany.

EU REP
MedNet EC-REP GmbH
Borkstrasse 10,

Mevoe Carmel Industrial Park, Israel.
3 Hazayit st., P.O.413
MEDITEX GROUP Ltd.

ENGLISH

Wrist Bandage

This Wrist Bandage reduces strain off the wrist and provides firm support while allowing normal daily functioning.

Supports the wrist joint and helps prevent injury.

Fits right or left hand.

Made of elastic Neoprene coated with synthetic towel top layer.

This product is recommended for use by individuals aged 18 and above only. For home, outpatient or institutional use. Single Patient Multiple Use. Medical Device.

Medical Indication

for the treatment of wrist pain due to strain, sprain, tendonitis and arthritis.

Provides universal fit and adjustability.

Caution

Consult your physician if you encounter severe sideeffects or discomfort. Use product for the mentioned indications only. For single patient use only. For external use only. Do not use product if you have allergy to one of it's components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider. Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Fitting Instruction

Fit around your wrist as shown in the picture. Secures the hook and loop fastener.

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

Material Composition

68% Neoprene, 16% Nylon (inner fabric), 16% Polyester (outer fabric).

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