Wrist Splint

REF 26

Ver.SW-26-250916

www.uriel-global.com

15815 SW 11th Court Rd. Ocala,
Florida 34473 USA

US Agent
FDAbasics LLC

5702 Niederlenz Switzerland

Lenzburgerstrasse 2

CH REP
Orthopartner AG

48163 Muenster Germany.

Borkstrasse 10,

EU REP
MedNet EC-REP GmbH

3 Hazayit st., P.O. 413
Mevo Carmel Industrial Park, Israel.

MEDITEX GROUP Ltd.

Wrist Splint

Fits left or right wrist. This splint is designed to provide firm wrist support by holding the wrist in a functional position of approximately 35 degrees of extension. An aluminum plate provides added rigidity.

This product is recommended for use by individuals aged 18 and above only.
For home, outpatient or institutional use.
Single Patient Multiple Use.
Medical Device.

Medical Indication

Ideally used for immobilization of the wrist following acute sprain, fracture, inflammation and carpal tunnel syndrome. Provides excellent support post cast removal and assists in reducing swelling around the wrist area.

Caution

Consult your physician if you encounter severe sideeffects or discomfort. Use product for the mentioned indications only. For single patient use only. For external use only. Do not use product if you have allergy to one of it's components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider. Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Fitting Instruction

This product is reversible and can be adjusted for left or right hand. To fit to the correct hand remove the aluminum bar from its pouch, flip it and insert it back. The convex side of the bar must face the inner side of your palm.

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

Material Composition

75% Polyester-spun Viscose,
25% Elastane double covered.

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