Wrist Splint

REF ST27R

Ver.SW-ST27R-250916

www.uriel-global.com

15815 SW 11th Court Rd. Ocala,
Florida 34473 USA

US Agent FDAbasics LLC

5702 Niederlenz Switzerland

Lenzburgerstrasse 2

CH REP Orthopartner AG

48163 Muenster Germany.

Borkstrasse 10,

EU REP MedNet EC-REP GmbH

Mevoe Carmel Industrial Park, Israel.

3 Hazayit st., P.O. 413

MEDITEX GROUP Ltd.

Wrist Splint

Provides firm wrist support whilst allowing normal daily functioning. With an aluminum palmar stay designed to provide firm wrist support by holding the wrist in a functional position at an extension of 35 degrees (adjustable, optional).

Made of elastic Neoprene coated with synthetic towel top layer.

This product is recommended for use by individuals aged 18 and above only. For home, outpatient or institutional use.

Single Patient Multiple Use. Medical Device.

Medical Indication

Recommended for the treatment of wrist pain due to strain, sprain, fracture, inflammation or Carpal Tunnel Syndrome, for management after plaster cast removal, treatment of ligament, tendon or muscle injuries.

Caution

Consult your physician if you encounter severe sideeffects or discomfort. Use product for the mentioned indications only. For single patient use only. For external use only. Do not use product if you have allergy to one of it's components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider. Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Fitting Instruction

Insert your hand into the splint with your thumb protruding through the hole as shown in the picture. Close and fasten the wrist band. Close and fasten the thumb band.

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

Material Composition

68% Neoprene, 16% Nylon (inner fabric), 16% Polyester (outer fabric).

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