Wrist Support

REF T25

Ver.SW-T25-250916

www.uriel-global.com

15815 SW 11th Court Rd. Ocala,
Florida 34473 USA

US Agent FDAbasics LLC

5702 Niederenz Switzerland

Lenzbürgetrasse 2
CH REP Orthopartner AG

48163 Muenster Germany.

EU REP MedNet EC-REP GmbH
Borkstrasse 10,

3 Hazayit st., P.O. 413
Mevo Carmel Industrial Park, Israel.
MEDITEX GROUP Ltd.

Wrist Support

This Wrist Support is made of thick THERMAL Neoprene, with 100% cotton towel lining. It helps retaining body heat which promotes faster healing and ease of pain. Relieves pressure from thumb joints. Takes the strain of the wrist and provides firm support whilst allowing for normal daily function. Supports wrist joints and prevents injury.

This product is recommended for use by individuals aged 18 and above only. For home, outpatient or institutional use. Single Patient Multiple Use. Medical Device.

Medical Indication

Treatment of wrist pain due to strain, sprain, tendonitis and arthritis.

Caution

Consult your physician if you encounter severe sideeffects or discomfort. Use product for the mentioned indications only. For single patient use only. For external use only. Do not use product if you have allergy to one of it's components. If you experience pain, swelling or sensation changes as a result of using this product, stop use immediately and consult your healthcare provider. Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Fitting Instruction

Put the brace on your hand with the thumb inserted in the thumb hole. Wrap around your wrist and close with the hook and loop fasteners for optimal support and comfort.

Hand wash only. Do not bleach. Do not iron. Do not expose to temperatures above 35°C / 95°F.

Storage and Transportation

Store in a clean, dry place, away from direct sunlight, excessive heat, and moisture. Avoid freezing. Transport under similar controlled conditions to prevent product damage.

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