# Relief Insert® Walker 2.0

# Relief Insert® Walker 2.0 Achill

## Relief Insert® Walker 2.0

EN Lower Leg /Foot Orthosis  
DE Unterschenkel-Fußorthese  
ES Ortesis de pierna y pie  
FR Orthèse de jambe-pied  
IT Ortesi per piede-gamba  
NL Onderbeen-voetorthese  
DA Underbens-fodortose  
SV Underben-fotortos  
PL Orteza podudzia i stopy  
SL Ortoza meča-stopalo  
HR Ortoza za goljenicu i stopalo  
PT Órtese de pé e perna inferior


## Relief Insert® Walker 2.0 Achill

EN Lower Leg /Foot Orthosis with  
Achilles tendon relief  
DE Unterschenkel-Fußorthese mit  
Achillessehnenentlastung  
ES Ortesis de pierna y pie con  
descarga del tendón de Aquiles  
FR Orthèse de jambe-pied avec  
décharge du tendon d'Achille  
IT Ortesi per piede-gamba con  
sgravio del tendine di Achille  
NL Onderbeen-voetorthese met  
ontlasting van de achillespees  
DA Underbens-fodortose med  
aflastning af akillesenen  
SV Underben-fotortos med avlast-  
ning för hälsenan  
PL Orteza podudzia i stopy z  
odciążeniem ścięgna Achillesa  
SL Ortoza meča-stopalo z razbre-  
menitvijo Ahilove tetive  
HR Ortoza za goljenicu i stopalo s  
rasterećenjem Ahilove tetive  
PT Órtese de pé e perna inferior  
com alívio do tendão de Aquiles

# DARCO

## Relief Dual® Plus

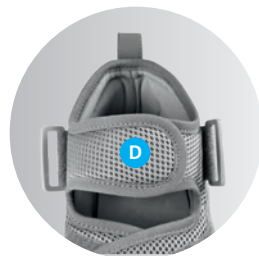## Relief Insert® 2.0

**E**

EN Strap loops  
 DE Umlenkschlaufen  
 ES Lazos de desviación  
 FR Boucles de sangles  
 IT Passanti  
 NL Doorhaallus  
 DA Stropper  
 SV Remöglå  
 PL Klamry do przewlekania  
 SL Zanke  
 HR Skretne omče  
 PT Fivelas

**H**

EN Valve for releasing the air from the air chambers  
 DE Ventil zum Ablassen der Luft aus den Luftkammern  
 ES Válvula para desinflar las cámaras de aire  
 FR Valve d'évacuation de l'air des alvéoles  
 IT Valvola per lo scarico dell'aria dalle camere d'aria  
 NL Ventiel waardoor lucht uit de luchtkamers kan ontsnappen  
 DA Ventil til uledning af luft fra luftkamrene  
 SV Ventil för att släppa ut luft ur luftkammarna  
 PL Zaworek do spuszczenia powietrza z klamer powietrznych  
 SL Ventil za izpuščanje zraka iz zračnih komor  
 HR Ventil za ispuštanje zraka iz zračnih komora  
 PT Válvula de saída do ar das câmaras de ar

**K**

EN Calf liner  
 DE Waden-Liner  
 ES Forro para pantorrilla  
 FR Doublure du mollet  
 IT Rivestimento del polpaccio  
 NL Kuitvoering  
 DA Læg-liner  
 SV Vadliner  
 PL Wkładka na tydzie  
 SL Vložek za meča  
 HR Podstava za list  
 PT Revestimento dos gêmeos

**F**

EN Forefoot strap  
 DE Vorfußgurt  
 ES Correa del antepié  
 FR Sangle d'avant-pied  
 IT Cinturino sull'avampiede  
 NL Voorvoetband  
 DA Forfødsrem  
 SV Framfotsrem  
 PL Pasek na przedniej części stopy  
 SL Trak za sprednj dio stopala  
 HR Pojas za prednji dio stopala  
 PT Correia de antepé

**I**

EN Calf splint with integrated air liner with air chambers  
 DE Waden-Schiene mit fest integrierten Air-Liner mit Luftkammern  
 ES Férula para pantorrilla con forro inflable integrado fijo y cámaras de aire  
 FR Attelle de mollet avec Air-Liner intégré et alvéoles  
 IT Stecca per polpaccio con Air-Liner integrato fisso e dotato di camere d'aria  
 NL Kuitbrace met vast geïntegreerde air-liner met luchtkamers  
 DA Læg-skinne med fast integreret Air-liner med luftkamre  
 SV Vadskenamed fast Air-Liner med luftkammare  
 PL Szyna tydkowa z umocowaną na stałe wkładką powietrzną z komorami powietrznymi  
 SL Opornica za meča s fikсно integriranim vložkom Air-Liner z zračnimi komorami  
 HR Udglaga za list s čvrsto ugrađenom zračnom podstavom sa zračnim komorama  
 PT Tala dos gêmeos com revestimento Air integrado de forma fixa com câmaras de ar

**L**

EN Air liner with air chambers  
 DE Air-Liner mit Luftkammern  
 ES Forro inflable con cámaras de aire  
 FR Air-Liner avec alvéoles  
 IT Air-Liner con camere d'aria  
 NL Air-liner met luchtkamers  
 DA Air-Liner med luftkamre  
 SV Air-Liner med luftkammare  
 PL Wkładka powietrzna z komorami  
 SL Air-Liner z zračnimi komorami  
 HR Zračna podstava sa zračnim komorama  
 PT Revestimento Air com câmaras de ar

**G**

EN Pump to fill the air liner air chambers  
 DE Pumpe zum Befüllen der Luftkammern des Air-Liners  
 ES Bomba para inflar las cámaras de aire del forro inflable  
 FR Pompe de gonflage des alvéoles de l'Air-Liner  
 IT Pompa per il riempimento delle camere d'aria dell'Air-Liner  
 NL Pomp voor het vullen van de luchtkamers van de air-liner  
 DA Pumpe til at fylde luftkamrene i Air-lineren  
 SV Pump för påfyllning av luftkammarna i Air-Liner  
 PL Pompka do napełniania komór powietrznych wkładki powietrznej  
 SL Črpalka za polnjenje zračnih komor na vložku Air-Liner  
 HR Pumpa za punjenje zračnih komora zračne podstave  
 PT Bomba para encher as câmaras de ar do revestimento Air

**J**

EN Automatic modification of the width: For unhooking, if necessary  
 DE Automatische Weitenregulierung: Im Bedarfsfall zum Aushaken  
 ES Regulación automática de la anchura: Se puede desenganchar en caso necesario  
 FR Réglage automatique de la largeur : Si nécessaire pour décrocher  
 IT Regolazione automatica della larghezza: Se necessario per sganciare  
 NL Automatische wijdeaanpassing: Waar nodig los te halen  
 DA Automatisk bredderegulering: Kan hægtes af efter behov  
 SV Automatisk viddreglering: kan hakas loss vid behov  
 PL Automatyczna regulacja szerokości: W razie potrzeby można rozpiąć  
 SL Samodejna regulacija širine: Za odpenjanje po potrebi  
 HR Automatska regulacija širine: Može se otkopčati po potrebi  
 PT Regulação automática em largura: Se necessário, para desenganchar

**M**

EN Front splint with front liner  
 DE Front-Schiene mit Front-Liner  
 ES Férula frontal con forro frontal  
 FR Attelle avant avec doublure avant  
 IT Stecca anteriore con rivestimento anteriore  
 NL Frontbrace met frontvoering  
 DA Front-skinne med Front-liner  
 SV Frontskena med frontliner  
 PL Szyna frontowa z wkładką frontową  
 SL Sprednja opornica s sprednjim vložkom  
 HR Prednja udglaga s prednjom podstavom  
 PT Tala dianteira com revestimento dianteiro

**Relief Insert® Walker 2.0** | Lower Leg / Foot Orthosis

**Relief Insert® Walker 2.0 Achill** | Lower Leg / Foot Orthosis with Achilles tendon relief

#### Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

The Relief Insert® Splinting System may only be used in combination with the Relief Dual® Plus Off-loading Shoe for walking. The splint is not designed for walking without a shoe.

There is a risk of thrombosis due to the restricted ankle movement.

#### Caution

There is a risk that the orthosis (splint) may break if the load is too heavy. Max. permitted load: 150 kg. The air chambers must not be over inflated to prevent pressure points on the leg and to prevent the air chambers from bursting. The air liner with pump cannot be washed.

## Purpose

**Relief Insert® Walker 2.0:** Lower leg / foot orthosis for immobilisation in a given position, as well as a foot-positioning orthosis

**Relief Insert® Walker 2.0 Achill:** Lower leg / foot orthosis and foot-positioning orthosis with Achilles tendon relief

## Design features

The lower leg / foot orthosis is available in five sizes with a universal fit for right and left.

The shoe can be customized to the left or right foot by individually adjusting the hook and loop straps. The lower leg / foot orthosis consists of the following components: One shoe (Relief Dual® Plus | Off-loading shoe with high ankle support), a splint (Relief Insert® 2.0 | Splinting System) with a grey air liner and integrated pump and a blue lining (consists of a forefoot strap, front and calf liner). The Relief Insert® Walker 2.0 Achill also includes heel wedges (DAW | DARCO Achilles Wedges).

The splinting system is used to position the lower leg / foot. In addition, a walker can be assembled from the splinting system using a therapeutic shoe to immobilise the lower leg / foot with simultaneous mobilisation. The exchangeable insole enables patients to individually adjust the fit.

**Relief Insert® Walker 2.0**

#### Indications

- > Arthrodesis of the first TMT joint (lapidus arthrodesis)
- > After metatarsal fracture surgery
- > After hallux valgus surgery
- > After hallux rigidus surgery
- > After hammer and claw toe surgery
- > Fractures of the phalanges
- > After surgery on the rearfoot
- > For ligament rupture of the ankle joint
- > Postoperative use after removal of Haglund's exostosis (with intact Achilles tendon)
- > Pre- and postoperative / post-traumatic immobilisation of the lower leg / foot, stable foot or ankle fractures, severe ankle sprains
- > Upper ankle joint arthrodesis

#### Contraindications

- > Acute Weber fractures (B, C)
- > Unstable fractures

**Relief Insert® Walker 2.0 Achill**

#### Indications

- > Achilles tendon ruptures
- > Postoperative use for Achilles tendon suture
- > Postoperative use after removal of Haglund's exostosis

#### Contraindications

- > Forefoot conditions
- > Plantar foot ulcers

## Combination with other products

- > PegAssist® Insole
- > Puzzle Insole®
- > PegContour® Insole (blue)
- > Relief Contour Insole
- > PegContour® Insole Uni
- > Toe Cap
- > DAW (DARCO Achilles Wedges)
- > Heel Wedges
- > Relief Insert® Walker 2.0 Lining

## Technical data

Sizes	Colour
XS 37.0 – 38.5	Grey-Blue
S 39.0 – 41.0	
M 41.5 – 43.0	
L 43.5 – 45.0	
XL 45.5 – 47.0	

#### Material

<b>Outsole:</b>	Rubber, EVA
<b>Insole:</b>	EVA
<b>Upper material of shoe:</b>	Polyester foam, microfibre
<b>Lining and forefoot strap:</b>	Polyester
<b>Hook and loop straps:</b>	Nylon
<b>Adhesive:</b>	Polyurethane glue
<b>Splints:</b>	Polyamide
<b>Pump:</b>	TPU
<b>DAW (heel wedges):</b>	EVA

## Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

To avoid the hook and loop straps getting in the way during the individual application steps, attach them behind onto the respective fleece side.

#### Using the Relief Insert® 2.0 as a splinting system:

When used as a splinting system, the Relief Dual® Plus Off-loading Shoe is not required. Set the shoe aside and proceed with opening the splinting system.

1. Fold the front splint away to the side.
2. If desired, another compatible DARCO insole can be placed in the splinting system. To do this, remove the grey insole. Then place and fix the desired insole analogously to the grey insole in the splinting system.
3. Place the leg on the calf liner of the calf splint. Make sure that the foot is in contact with the insole.
4. The width of the guide splint automatically adjusts to the patient's needs by means of the adjuster on the back of the guide splint.
5. Once the leg is positioned, close the splinting system. To do this, place the front splint on the shin. Make sure that the front splint is positioned centrally within the calf splint. Guide the three hook and loop straps through the loops. Pull the hook and loop straps as tight as necessary. Ensure that the leg is supported comfortably. Optionally, the blue forefoot strap (F) can be attached to the forefoot for additional support.

#### Using the Relief Insert® Walker 2.0 as a lower leg/foot orthosis:

Perform steps 1 – 5 as described. After that, please continue as follows.

6. Let the patient step into the opened foot section of the Relief Dual® Plus Off-loading Shoe with the orthosis (splint) on. For particularly easy entry, open the Relief Dual® Plus completely using the hook and loop straps (A, B, C). Make sure that the insole is positioned securely at the front in the shoe.
7. Now close the off-loading shoe with the two hook and loop straps C of the heel cap.
8. The inner shoe tabs can be placed over each other either way (D).
9. Close the two hook and loop straps A, B as described below. Fold the strap A on the outer side (left or right) around the tab and secure it firmly. Close the shoe with the strap on the other side. If needed, the length of the upper hook and loop strap A can be customised to fit. Now close the shoe with the lower hook and loop fastener B such that the DARCO logo is on the outside of the shoe.
10. To achieve additional stabilisation of the ankle joint, the volume of the air chambers can be increased using the pump attached to the side. To reduce the volume, release the air using the silver valve. This allows for individual filling and adjustment. Ensure that the leg is supported comfortably. If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers using the valve. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

#### Using the Relief Insert® Walker 2.0 Achill as an Achilles walker:

11. Fold the front splint to the side.
12. Leave the insole in the splint.
13. Place the wedges in the heel area of the splint on the insole, corresponding to the degree of plantar flexion that is therapeutically required. A wedge has a height of approx. 11 mm. Depending on the number of wedges used, the height varies and the angle of plantar flexion can, thus, be adjusted individually. 3 wedges are equivalent to approx. 120° slant.
14. Fix the wedges to the insole using the adhesive dots.
15. Depending on the therapy course, the wedges can be removed and, thus, the height adjusted.

Once the wedges are positioned, please carry out steps 3 to 10.

#### Changing the lining

##### Removing the lining:

16. Remove the blue calf liner attached to the calf splint with hook and loops.
17. The grey air liner with integrated air chambers and pump remains in the product and must not be removed.
18. Now carefully remove the blue front liner attached to the front splint with hook and loop fasteners.
19. The calf liner, front liner and forefoot strap can be washed, if required.

##### Fitting the lining:

After cleaning the blue lining, please carry out the steps described below:

20. First attach the front liner to the front splint using the hook and loop fasteners. To do this, fold the liner in the middle lengthwise and place it centrally (according to the shape of the front splint) on the front splint. By unfolding the liner, it clings to the front splint in the correct position. Gently press the front liner against the splint.
21. The calf liner is also folded lengthwise in the middle for attachment to the splint. Place the calf liner in the centre of the calf splint with the notches at the level of the ankle. By unfolding the liner, it clings to the calf splint in the correct position. Gently press the calf liner.

To close the splinting system or walker, follow the steps described from step 1.

#### Instructions for use

The Relief Insert® Splinting System may only be used in combination with the Relief Dual® Plus Off-loading Shoe for walking. The splint is not designed for walking without a shoe.

## Cleaning instructions

### Washing

#### Shoe, splint, grey insole, wedges:

If these are slightly soiled, wipe with a damp cloth or clean with commercially available disinfectants.

#### Liners:

To wash, the blue calf and front liners are removed from the splint and then washed at approx. 60° C in a washing machine. The optional forefoot strap can also be washed at 60° C. The air liner with air chambers and the pump cannot be washed.

#### Drying

Do not dry any of the components of the Relief Insert® Walker 2.0 | Achill in a dryer or on a radiator, but dry them at room temperature.

## Maintenance

The product does not require maintenance.

## Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning.

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