Urias® Air Splints

Non-Sterile Air Splints Instructions for Use

Arden Medical

Manufactured By:

Arden Medical Ltd, Arden Road, Arden Forest Industrial Estate, Alcester, Warwickshire B49 6HN, England
www.arden-medical.com

Non Sterile Air Splint

Instructions for use

The non-sterile air splint range, are designed to be used as an aid to rehabilitation by trained therapists.

1. A thin cotton sleeve should cover the patients limb while the splint is in use.
2. The splint must be orally inflated. Pressure should not exceed 40mm Hg and should be read when limb is at rest.
3. The splints may be wiped over with a mild disinfectant to clean them, then dried with a towel. If necessary, the valve may be disconnected and washed and the inflation tube cleaned with a test tube brush dipped in mild disinfectant, allowed to dry and then re-assembled.

DO NOT LET FLUID RUN INSIDE THE SPLINT AS IT CAN NOT DRAIN OUT.

Accessories

Mouthpiece - Ref 75 000 0 - Personal detachable mouthpiece.
Filter Bottle - Ref 75 011 0 - Contains crystals which absorb excess moisture produced during inflation.

Precautions

1. The splint should not be worn in direct sunlight.
2. No splint should be left on for more than one hour.
3. Do not use the tube as a handle when fitting the splint.
4. Do not store in sub zero temperatures.
Recommended storage temperature is +10°C.
5. Dispose of in household rubbish bin.
6. Do not burn on household fire or electric incinerator.

For further information on Non-Sterile Air Splints see relevant user guide.

Symbol Explanation

Attention, see Instructions
for use

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