
Finger splint (DIP+PIP) - Malleable Ref_10

Fingerschiene (DIP+PIP) - Verformbar
Attelle de doigt (IPD+IPP) - Souple
Stecca per dita (DIP+PIP) - Malleabile
Férula de dedo (AID+AIP) - Maleable
Ujsín (DIP+PIP) - Hajlítható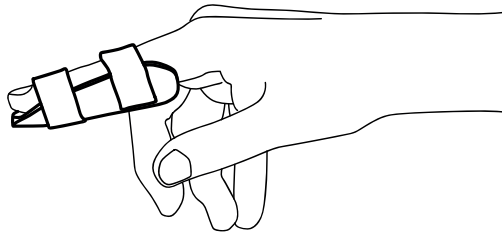

		PREMIUM		STANDARD	
		XXXS-S	L-XL	XXXS-S	L-XL
A	Product shell	1	1	1	1
D	Textile strap - 60x15 mm	2	1		
D	Textile strap - 90x15 mm		2		
F	Adhesive tape - 80x15 mm			2	3

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Instructions for use - EN

INDICATION

Recommended for fixation of the PIP and DIP joints following injuries, stress and strain, ligament injuries and certain fractures or in case of degenerative states/ arthritis (also rheumatoid arthritis, gout, autoimmune diseases) or as part of post surgical rehabilitation in certain cases. The malleable Chrisofix® long finger splint fixates the finger in an appropriate and desirable position - if this differs from the above described anatomical (0° or straight) position, while thanks to its innovative design it also ensures stability, comfort and precise fitting.

PRODUCT VERSIONS

PREMIUM: Product with textile straps
PREMIUM FORTE: Premium product extended with aluminium containing strap
SHELL: Premium product without any straps, just the product shell
DISINFECTABLE SHELL: Shell with special foil on both side so it allows disinfection
STANDARD: Corrugated aluminium core embedded in cotton-laminated foam (inside) and polyethylene foam covered with polyethylene foil (outside)

PRODUCT CHARACTERISTICS

Chrisofix® orthoses can ensure limb fixation according to diverse clinical requirements. Chrisofix® products are repeatedly adjustable, yet shape-retaining, are light and washable. They limit the movement of the limb only to the necessary degree, but when used with the corresponding strap they also immobilise the limb like a POP or thermoplastic splint. It is clinically proven that the application of Chrisofix® orthoses results in pain reduction and improvement of functioning capacity.

APPLICATION

Chrisofix® orthoses are supplied in a pre-shaped form ready for their application. Place the limb into the orthosis according to the illustration, and adjust the orthosis with gentle pressure. Finally, fix the limb inside the orthosis with the straps supplied.
Shell and Disinfectable versions: To be completed with a bandage.

MATERIAL AND PRODUCT CARE

Chrisofix® orthoses are made of a non-allergenic and phthalate-free sandwich composite consisting of polyethylene foam, polyamide, cotton, and corrugated and perforated aluminium. The layer that has direct contact with the skin is skin-friendly and soft, so there is no need for extra cushioning. The structure is breathable, repeatedly adjustable without heat or water, shape-retaining and X-ray transparent. The orthosis and its accessories (straps, lining) are non-sterile and can be hand-washed at 30°C with commercially available detergent. Dry and store at room temperature. None of our products contain any hazardous substances. Once used, please discard as household waste.
Disinfectable versions: It is disinfectable and should be left to air dry.

SAFETY PRECAUTIONS

The orthosis should be applied to clean, undamaged skin or following wound treatment. Avoid an overly tight application or one that leaves mark on the skin. The first application and major changes to the form of the orthosis should be performed by professionals. Excessively frequent shaping / bending of the orthoses should be avoided as this may reduce the stability of the aluminium core. Chrisofix® orthoses are not suitable for heavy physical work. Patients should avoid any activity that may lead to the orthosis getting damaged, broken, wet, contaminated or having a major change in form. Failures resulting from misuse are not covered by the product warranty. Do not use a broken or damaged orthosis. Do not cut the product! Avoid the combination with other fixation products on the same limb, except in case of medical instruction.
Chrisofix® products are produced according to the quality control system ISO:13485. However, should arise any complain regarding the product, please contact your local representative dealer. Please report any serious incident that has occurred in relation to the device to the competent authority of the Member State in which the user and/or patient is established, the manufacturer and the distributor. Please keep the packaging and Instructions for Use until the end of the treatment. The product has no expiration date.

ACCESSORIES

A- product shell, B- lining, C- aluminium strap, D- textile strap, E- external stiffener, F- adhesive tape, G- Textile band with neck guard. For the exact parts list, see the table on the first page of the instructions.

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