## Orthosis for paresis (wrist & hand & thumb)

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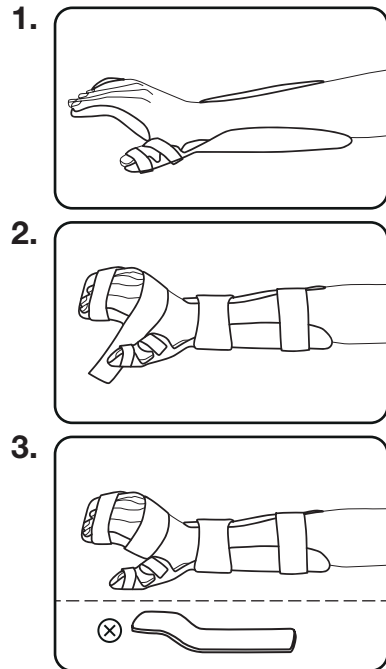
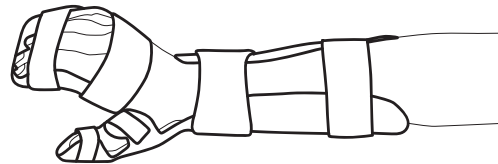
## Instructions for use - ENG

### INDICATION

Fixation or immobilisation of the wrist, hand, long fingers and thumb after injuries (e.g. distorsion), surgery and fixation with plaster of Paris. Conservative treatment in tendonitis, inflammation and degenerative diseases of the wrist, hand, long fingers and thumb even as resting splint. Rehabilitation phase after hand surgery. Recommended for patients with paresis / spasticity of the wrist, long fingers and thumbs.

### PRODUCT VERSIONS

**PREMIUM:** Product with textile straps  
**PREMIUM FORTE:** Premium product extended with aluminium containing strap  
**SHELL:** Premium product without any straps, just the product shell  
**DISINFECTABLE SHELL:** Shell with special foil on both side so it allows disinfection  
**STANDARD:** Corrugated aluminium core embedded in cotton-laminated foam (inside) and polyethylene foam covered with polyethylene foil (outside)

### PRODUCT CHARACTERISTICS

Chrisofix® orthoses can ensure limb fixation according to diverse clinical requirements. Chrisofix® products are repeatedly adjustable, yet shape-retaining, are light and washable. They limit the movement of the limb only to the necessary degree, but when used with the corresponding strap they also immobilise the limb like a POP or thermoplastic splint. It is clinically proven that the application of Chrisofix® orthoses results in pain reduction and improvement of functioning capacity.

### APPLICATION

Chrisofix® orthoses are supplied in a pre-shaped form ready for their application. Place the limb into the orthosis according to the illustration, and adjust the orthosis with gentle pressure. Finally, fix the limb inside the orthosis with the straps supplied.  
Shell and Disinfectable versions: To be completed with a bandage.

### MATERIAL AND PRODUCT CARE

Chrisofix® orthoses are made of a non-allergenic and phthalate-free sandwich composite consisting of polyethylene foam, polyamide, cotton, and corrugated and perforated aluminium. The layer that has direct contact with the skin is skin-friendly and soft, so there is no need for extra cushioning. The structure is breathable, repeatedly adjustable without heat or water, shape-retaining and X-ray transparent. The orthosis is not a sterile device. It is washable at 30 °C and should be left to air dry. Store at room temperature. None of our products contain any hazardous substances. Once used, please discard as household waste.  
Disinfectable versions: It is disinfectable and should be left to air dry.

### SAFETY PRECAUTIONS

The orthosis should be applied to clean, undamaged skin or following wound treatment.

Avoid an overly tight application or one that leaves mark on the skin. The first application and major changes to the form of the orthosis should be performed by professionals. Excessively frequent shaping / bending of the orthoses should be avoided as this may reduce the stability of the aluminium core. Chrisofix® orthoses are not suitable for heavy physical work. Patients should avoid any activity that may lead to the orthosis getting damaged, broken, wet, contaminated or having a major change in form. Failures resulting from misuse are not covered by the product warranty. Do not use a broken or damaged orthosis. Do not cut the product! Avoid the combination with other fixation products on the same limb, except in case of medical instruction.

Chrisofix® products are produced according to the quality control system ISO:13485. However, should arise any complain regarding the product, please contact your local representative dealer. Please report any serious incident that has occurred in relation to the device to the competent authority of the Member State in which the user and/or patient is established, the manufacturer and the distributor. Please keep the packaging and Instructions for Use until the end of the treatment. The product has no expiration date.

### ACCESSORIES

A- product shell, B- lining, C- aluminium strap, D- textile strap, E- external stiffener, F- adhesive tape, G- Textile band with neck guard. For the exact parts list, see the table on the first page of the instructions.

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	PREMIUM		PREMIUM WITH LINER		PREMIUM FORTE		PREMIUM FORTE WITH LINER		PREMIUM FORTE PLUS WITH LINER		BESPOKE / COUNTRY SPECIFIC		SHELL		DISINFECTABLE SHELL WITH LINER		
	XS	S	M	L	XS	S	M	L	XS	S	M	L	XS	S	M	L	XS-M
A Product shell	1	1	1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
B Lining																	
C Aluminium strap					1	1	1	1	1	1	1	1					
E External stiffener																	
D Textile strap - 60X15 mm			2	2	2	2	2	2	2	2	2	2	2	2	2	2	2
D Textile strap - 150X25 mm			1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
D Textile strap - 180X25 mm			1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
D Textile strap - 220X25 mm			1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
D Textile strap - 260X25 mm			1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
D Textile strap - 180X38 mm			2	2	2	2	2	2	2	2	2	2	2	2	2	2	2
D Textile strap - 220X38 mm			2	2	2	2	2	2	2	2	2	2	2	2	2	2	2
D Textile strap - 240X38 mm			1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
D Textile strap - 260X38 mm			1	1	1	1	1	1	1	1	1	1	1	1	1	1	1
D Textile strap - 280X38 mm																	

Width of palm Handbreite	Size Größe
Largeur de la paume Ampezia del palmo Anchura de la palma Tenyérszélesség <td>Taille Taglia Talla Méret</td>	Taille Taglia Talla Méret
	< 6 cm XS
	6-8 cm S
	8-9 cm M
	> 9 cm L

Chrisofix AG  
Dorfstrasse 15.,  
8302 Kloten, Switzerland  
Tel.: +41 52 6701160 | hello@chrisofix.ch  
www.chrisofix.com | EN ISO 13485:2016

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