Mallet finger splint (DIP)

Ref_12


Fingerendgelenkschiene (DIP)

Attelle de doigt maillet (IPD)

Stecca per dito a martello (DIP)

Férula para dedo en martillo (AID)

Végperc sín (DIP)

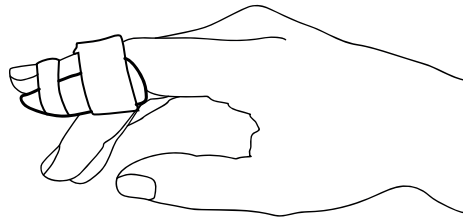		PREMIUM
		S-L
A	Product shell	1
D	Textile strap - 60x15 mm	1
D	Textile strap - 60x7,5 mm	1

Length of the splint Schienenlänge	Size Größe
Longueur de l'attelle Taille	
Lunghezza della stecca Taglia	
Longitud de la férula Talla	
A sin hossza Méret	
3 cm	S
4 cm	M
5 cm	L

Instructions for use - ENG

INDICATION

In certain cases for injuries or degenerative states/arthritis, fractures of the fingertips, injuries of the extensor tendons of the fingertips, arthritis or pain in the end joint of the finger. Thanks to the innovative design of the Chrisofix® mallet finger splint they secure appropriate positioning and stability while ensuring comfort, freedom of the neighboring joints and precise fitting.

PRODUCT VERSIONS

PREMIUM: Product with textile straps

PREMIUM FORTE: Premium product extended with aluminium containing strap

SHELL: Premium product without any straps, just the product shell

DISINFECTABLE SHELL: Shell with special foil on both side so it allows disinfection

STANDARD: Corrugated aluminium core embedded in cotton-laminated foam (inside) and polyethylene foam covered with polyethylene foil (outside)

PRODUCT CHARACTERISTICS

Chrisofix® orthoses can ensure limb fixation according to diverse clinical requirements. Chrisofix® products are repeatedly adjustable, yet shape-retaining, are light and washable. They limit the movement of the limb only to the necessary degree, but when used with the corresponding strap they also immobilise the limb like a POP or thermoplastic splint. It is clinically proven that the application of Chrisofix® orthoses results in pain reduction and improvement of functioning capacity.

APPLICATION

Chrisofix® orthoses are supplied in a pre-shaped form ready for their application. Place the limb into the orthosis according to the illustration, and adjust the orthosis with gentle pressure. Finally, fix the limb inside the orthosis with the straps supplied.

Shell and Disinfectable versions: To be completed with a bandage.

MATERIAL AND PRODUCT CARE

Chrisofix® orthoses are made of a non-allergenic and phthalate-free sandwich composite consisting of polyethylene foam, polyamide, cotton, and corrugated and perforated aluminium. The layer that has direct contact with the skin is skin-friendly and soft, so there is no need for extra cushioning. The structure is breathable, repeatedly adjustable without heat or water, shape-retaining and X-ray transparent. The orthosis is not a sterile device. It is washable at 30 °C and should be left to air dry. Store at room temperature. None of our products contain any hazardous substances. Once used, please discard as household waste.

Disinfectable versions: It is disinfectable and should be left to air dry.

SAFETY PRECAUTIONS

The orthosis should be applied to clean, undamaged skin or following wound treatment.

Avoid an overly tight application or one that leaves mark on the skin. The first application and major changes to the form of the orthosis should be performed by professionals. Excessively frequent shaping / bending of the orthoses should be avoided as this may reduce the stability of the aluminium core. Chrisofix® orthoses are not suitable for heavy physical work. Patients should avoid any activity that may lead to the orthosis getting damaged, broken, wet, contaminated or having a major change in form. Failures resulting from misuse are not covered by the product warranty. Do not use a broken or damaged orthosis. Do not cut the product! Avoid the combination with other fixation products on the same limb, except in case of medical instruction.

Chrisofix® products are produced according to the quality control system ISO:13485. However, should arise any complain regarding the product, please contact your local representative dealer. Please report any serious incident that has occurred in relation to the device to the competent authority of the Member State in which the user and/or patient is established, the manufacturer and the distributor. Please keep the packaging and Instructions for Use until the end of the treatment. The product has no expiration date.

ACCESSORIES

A- product shell, B- lining, C- aluminium strap, D- textile strap, E- external stiffener, F- adhesive tape, G- Textile band with neck guard. For the exact parts list, see the table on the first page of the instructions.

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