EN Instructions for use

PegContour® Insole Uni | Off-loading Cushion Insole

Important notes: If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

We recommend that you do not drive a vehicle while you are wearing this aid in a special shoe. Take extreme caution and walk more slowly than usual. This aid is intended for single-patient use only.

Purpose

The PegContour® Insole Uni is a three-dimensional insole system for redistribution of plantar pressure as well as selective off-loading.

Indications / Contraindications

- Indications**
- > For use following hallux valgus and small toe surgeries (Austin, Akin, Scarf, Weil, Hohmann, etc.)
 - > Angioneuropathic changes of the foot in combination with mild foot deformities
 - > Healing of ulcers in the heel area (heal foot ulcers)
 - > Heel spurs (plantar fasciitis)
 - > Heel wounds
 - > Postoperative use and treatment of the heel
- For selective off-loading for:
- > Plantar lesions
 - > Plantar ulcers / ulcer prevention
 - > Pressure points
- Contraindications**
- > Marked foot deformities of the diabetic foot

Design features

The PegContour® Insole Uni, made of 100% from PU material, provides perfect pressure redistribution. The lengthwise and crosswise sole covering provides a high level of wearing comfort. Easy removal of individual pegs enables targeted relief.

Use

1. Remove the grey or black original sole from the shoe (fig. 1).
2. Determine the area of the foot that needs relief (fig. 2).
3. Carefully remove the pegs at the site at which the pressure should be relieved (fig. 3). Use a suitable tool for this or remove the pegs by hand.
4. Reinsert the adapted PegContour® Insole Uni in the shoe with the pegs facing downwards (fig. 4).
5. Fitting finished.

Note: Signs of fatigue can occur in the cover layer when there is a large gap.

Combination with other products

- > Sole with blue pegs for: MedSurg, MedSurg Pro®, Relief Dual®, Relief Dual® Plus, Relief Insert® Walker 2.0
- > Sole with grey pegs for: AllRound Shoe®
- > Sole with black pegs for: OrthoWedge, OrthoWedge Light, HeelWedge®, Commodus Open®, SlimLine®

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Technical data

- Sizes**
- > Soles with blue pegs available in 7 sizes
 - > Soles with grey pegs available in 6 sizes
 - > Soles with black pegs available in 5 sizes
 - > Universal fit for right and left
- Colour**
- Blue, Grey, Black
- Material**
- The insole is made of polyurethane. The sole covering is made of polyester.

Maintenance

The product does not require maintenance.

Cleaning instructions

The soles can be cleaned with lukewarm water and soap or with conventional disinfectants. Do not dry the sole on a heating unit; instead allow to dry at room temperature.

Disposal

Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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ALBO-Healthcare GmbH